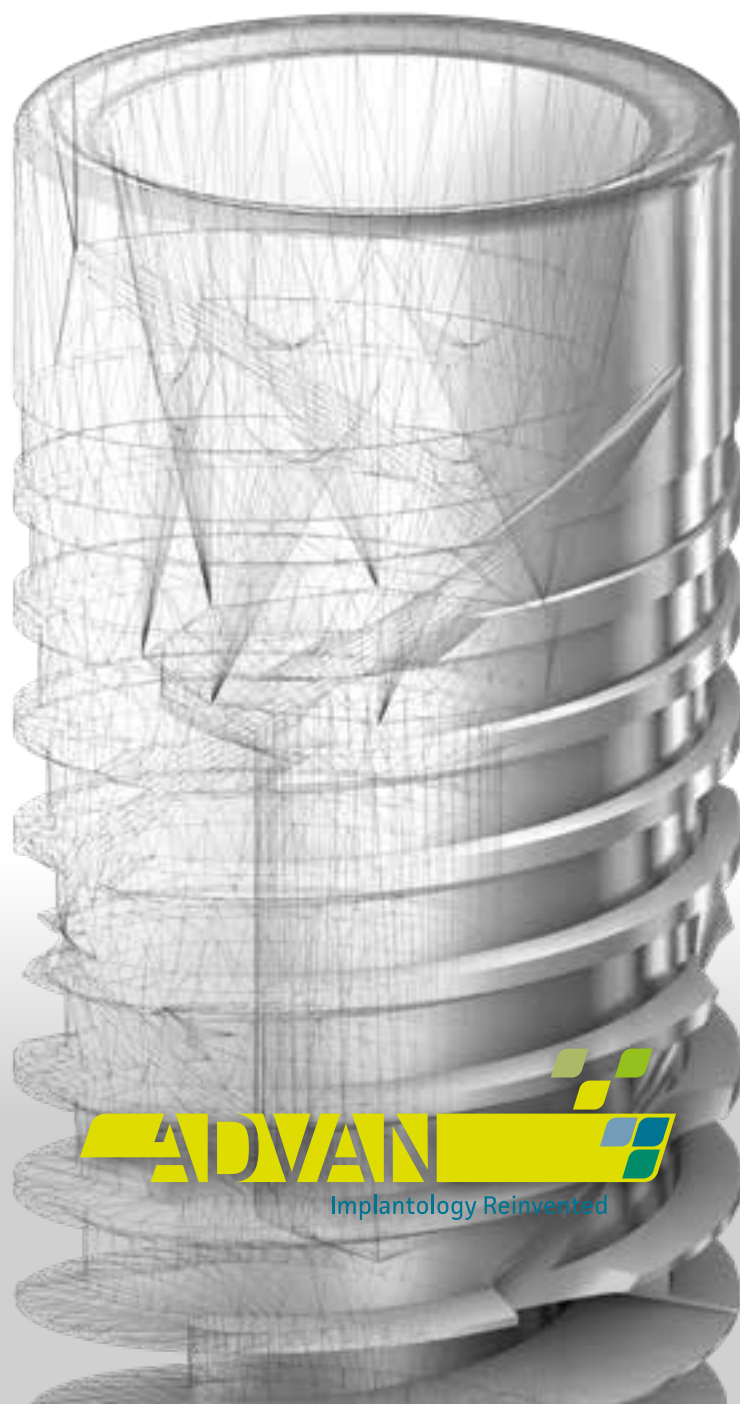
PRODUCT CATALOGUE

# GTB

THE FIRST HIGH BIOLOGICAL PERFORMANCE IMPLANT SYSTEM

<b>About us</b>	4
<i>Eccellenza in implantologia dentale</i>	
<i>Excellence in dental implantology</i>	
<i>Excelencia en implantología dental</i>	
<hr/>	
<b>GTB</b>	8
<i>Un nuovo progetto in implanto-protesi</i>	
<i>A new project in dental implants</i>	
<i>Un nuevo proyecto en implantes dentales</i>	
<hr/>	
<b>GTB</b>	22
<i>Impianti</i>	
<i>Fixtures</i>	
<i>Implantes</i>	
<hr/>	
<b>GTB Tzero</b>	26
<i>Impianti</i>	
<i>Fixtures</i>	
<i>Implantes</i>	
<hr/>	
<b>Confezionamento</b>	30
<b>Packaging</b>	
<b>Embalaje</b>	
<hr/>	
<b>Chirurgia</b>	34
<b>Surgery</b>	
<b>Cirugía</b>	
<hr/>	
<b>Strumenti chirurgici</b>	43
<b>Surgical tools</b>	
<b>Herramientas quirúrgicas</b>	
<hr/>	
<b>Protesi</b>	48
<b>Prosthesis</b>	
<b>Prótesis</b>	
<hr/>	
<b>Magellan</b>	88
<i>Chirurgia guidata</i>	
<i>Guided surgery</i>	
<i>Cirugía guiada</i>	
<hr/>	
<b>Descrizione dei simboli</b>	98
<b>Symbol description</b>	
<b>Descripción de símbolos</b>	
<hr/>	

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eng

**Advan** pursues an innovative vision of implantology: the implant system is placed at the center of the digital workflow as an essential requirement for any dental team in order to meet at best the clinical needs of patients.

With this purpose, **Advan** is the result of the union of two companies with more than 25 years of experience in design, production and sale of dental implant products and with the aim of distributing their products in Italy and worldwide.

**Advan** can fully follow the development of a medical product, from concept to engineering, from mechanical tests to management of clinical trials up to certification and mass production.

This certification is issued for the total quality system, all activities are therefore subject to the procedures provided by the quality manual, the production processes are constantly monitored to ensure the quality and safety of the products and their traceability.

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Operational activities are carried out in one production sites: precision machining at the plant, exclusively dedicated to implantable medical products, equipped with the most up-to-date CNC machines for providing a product of superior precision, strictly duplicated for every lot and controlled with the most advanced systems of metrology (laser technology); surface treatments, assembly of semi-finished products, marking, decontamination and packaging are carried out at the production area equipped with a certified cleanroom for the production of implantable medical products (ISO 7 Class Clean Room).

**Advan** also provides its know-how and operative structures for companies or professionals who want to carry out their projects, offering all the necessary resources and support from the development of prototypes to production.

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*We link the clinical needs of the dental team with the scientific research and the industry technology*

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In **Advan** the experience is combined with the constant and meticulous attention concerning the materials development, the passion for innovation, the most up-to-date production technologies and the clinical guidelines, supported by the scientific contribution of our Opinion Leaders.

#### Mission

To study and to generate implant systems that are easy to use, with high clinical performances and highly predictable long term results, also through the digital workflow, are the clear and defined objectives that everyone, in **Advan**, knows and constantly pursues.

**Advan** is committed to deliver significant value to Patients and to Professionals, everyone is personally responsible and aware that quality of products has an impact on the quality of life for thousands of lives every day.

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## Fields of application of Advan solutions

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eng

**Over the past 20 years many changes occurred in the field of dental implants and concepts that seemed to have become axioms were revised.** After having focused almost exclusively on the concept of osteointegration, recently other key factors have been studied: the maintenance of perimplant bone tissue, especially at the crestal level, the improvement of implant system functionality after loading, i.e. the biomechanics of the implant. The new approach includes accurate evaluation of the coupling between implant, abutment and other prosthetic components after osseointegration. In this area, sometimes accidentally and some other times voluntarily, new features of the implantable osteointegrated systems have been developed.

The study of couplings implant-abutment has highlighted how some geometries ensure a better seal of the implant-abutment interface with respect to the bacterial and fungal colonization, thus preventing the formation of bacterial plaque inside the implant screws and at the implant-bone interface [1][13]. The implant-abutment connections can be of fundamental importance in preventing the micromotion of the abutment in the implant after loading, from a mechanical point of view (i.e. transmission of off-axis and lateral forces to the implant and to the bone, with overload risk) and from a biological point of view (i.e. increasing of microgap implant-abutment, with possible microbiological colonization and inflammation of the perimplant soft tissues during loading). It has been noted, besides, that the presence of a platform switching or platform shifting permits to better transfer the load between implant, abutment and bone and at the same time moves away from the crestal bone portion the microbiological risk at implant-abutment connection level.

The presence of the platform shifting, associated with the surgical subcrestal placement and the rough surface of the implant up to the implant-abutment interface, allows the bone to grow above the platform itself and hence to obtain a dental implant system with a better biomechanical efficiency [2][19][20][22]. Additionally the presence of a concave profile of the abutment ensures a greater amount of soft tissue, a better system protection and a lower bone loss risk, because it works as an **integrated biological system** [3]. In this scientific context and updated point of view, the new implant system GTB has been designed and developed to refine and optimize the above concepts. The GTB implant screw macrogeometry has been designed to allow a controlled and progressive load transfer in order to transfer greater compressive forces and to reduce tensile forces since the bone withstands the compression much better than the traction [4]. The thread gradually widens towards the rounded and atraumatic apex of the implant with a double reverse buttress thread which facilitates the surgical insertion. An helix discharge incision helps removing the

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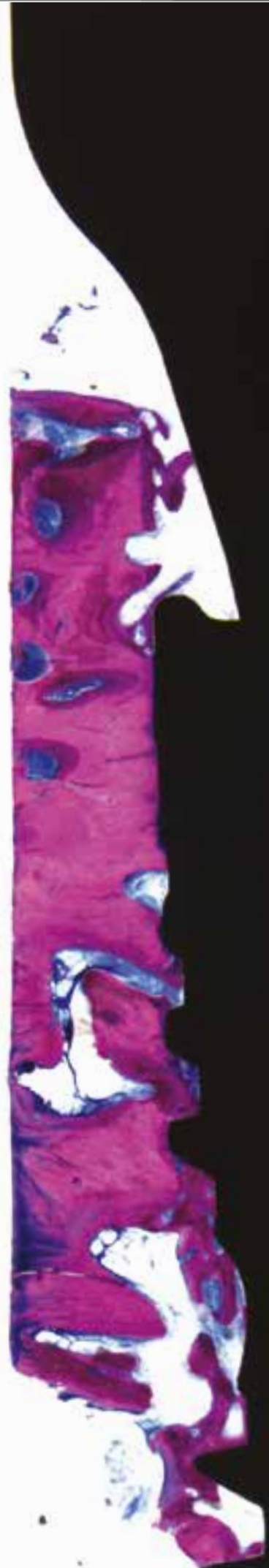
fluids that could cause an excessive compression during insertion and allows for less invasive osteotomies while allowing wetting of the entire implant surface using only mechanical arrangements. The implant crest module prevents cortical bone compression and the platform shifting, with a complex and innovative profile both horizontal and vertical, increases the active surface of the platform itself. The connection between the implant and the abutment is conical, with effective control of the microgap and without micromotion between the implant and the abutment. The presence of an octagonal index facilitates the accomplishment of all prosthetic phases, without any active role in maintaining stability of the implant-abutment connection (reducing risk of abutment loosening). The same prosthetic platform for all diameters, allows to easily perform all rehabilitation techniques; the abutments are designed with a concave profile that improves the maintenance of the soft tissues and guarantees a remarkable aesthetic result. The GTB system is designed with the intent to increase the biomechanical performance of the implant system with an improvement in terms of predictability, stability and aesthetics. All the claims have been scientifically proved by FEA analysis, lab tests and clinical evidence.

#### Osteointegrated system with high biological performance

The implantable GTB system is a **biologically high performance** osteointegrated system, because of the minimally invasive bone screws dimensions that allows the preservation of biological tissues after the placement of the implant and the abutment. The system is characterized by a controlled and gradual load distribution through double threads with gradual depth and profile, which allows the distribution of the forces with a gradual increase along the major axis of the system, reducing the stress at cortical bone level. The implant design is a geometrically complex horizontal and vertical platform shifting, with a unique prosthetic platform for all the diameters. The connection is a 11° cone with octagonal non rotational index. The conical coupling ensures maximum reduction in the microgap abutment-implant interface and a significant increase in the stability of the prosthetic component under load avoiding any abutment-implant micromotion. The octagonal index eases prosthetic procedures with eight positions prosthetic index. The concave emerging profile of prosthetic components al-

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low a significant increase in the amount of peri-implant connective tissue and the protection of the implant-prosthetic connection [17].

#### Characteristics of the bone screw

The implant screws GTB have been designed with the aid of the FEA (Finite Element Analysis) to transmit the masticatory load progressively along the major axis of the implant, by using most of all the apical third and not loading as much as possible the implant coronal third. This choice is driven by the need to avoid overload on the cortical bone, so as to preserve it, and to maintain the hard and soft tissues levels. Moreover its design has been focused on reducing the tensile forces transmitted to the bone tissue, both in their average value and in the load peaks. This choice derives from the biology of the bone tissue: it is strongest when loaded in compression, 30% weaker when subjected to tensile forces and 65% weaker when loaded in shear [4]. The GTB implant screws feature a cylindrical walled thread with a tapered inner diameter and a modified reverse buttress thread: the variable thread section allows to increase the compression forces, reducing the tensile, according to the best loading conditions for the bone. The threads have a triangular profile, aggressive in the apical third portion of the implant, and trapezoidal in the coronal third portion to minimize localized overloads due to the sharp corners (typical of V shaped thread). Finally, the thread section is reverse buttress with faces angle 10/30 to limit the thread dimensions increasing the bone tissue, to have a thread with a better penetration in the bone tissue and ensure a limited tensile component transmitted to the bone. Ensuring greater volume available for the bone tissue means having the same occlusal load distributed on a larger volume of bone, improving the biomechanics of the implant screw even in case of reduced diameters or short implants [19]. The double thread makes the insertion easier and ensures an excellent primary stability even with compromised bone densities. The pitch is 0.6 mm. The GTB implant screws have a fluids discharge U-shaped notch, allowing the flow of liquids during implant screwing in the bone site. The presence of a fluids discharge notch results from the design choice to have surgical instruments for the osteotomy congruent with the implant screw (drills prepare the implant site as deep as implant length), thus avoiding the hydrostatic compression of

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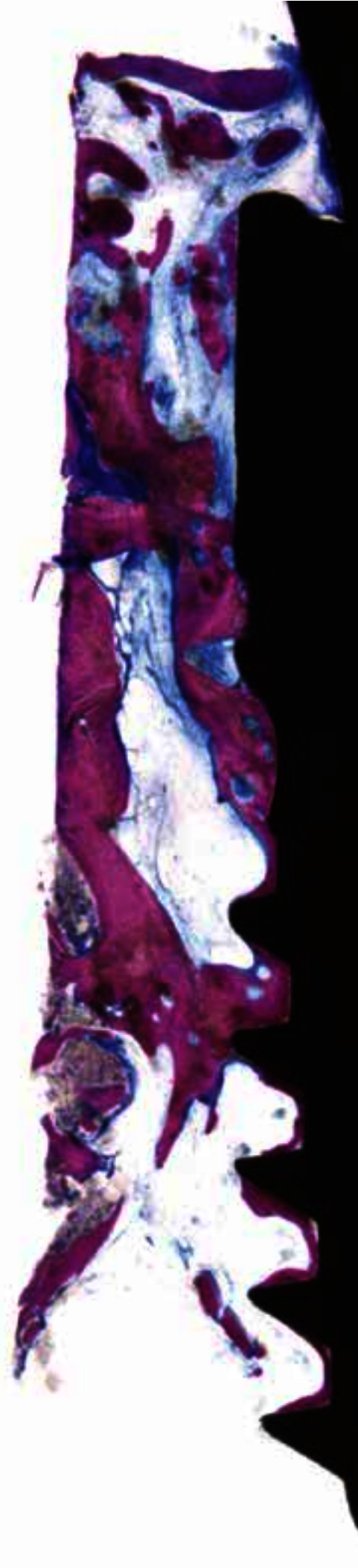
GTB IS DESIGNED FOR IMMEDIATE LOADING

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the clot at the apex of the implant site. The implant apex is convex and atraumatic to safely perform mini sinus lift techniques with the maximum respect of the Schneiderian membrane. Looking at the FEA results, it becomes evident how the GTB implant transfer the occlusal load to the bone mostly through its apical portion, minimizing the tensile component. It is moreover important to note that shortest implants have a biomechanical behaviour virtually identical to implants with length of 9.0 mm and same diameter; additionally a small increment in diameter, switching from implants with diameter 3.6 mm to implants with diameter 4.3 mm, considerably increases the biomechanical efficiency of GTB implant, halving the stress values transmitted to the bone tissue. The subcrestal placement of the implant screws is a surgical indication that allows the growth of the cortical bone above the platform, making the GTB implants biomechanically even more efficient in order to protect the cortical bone from overloading and especially from the tensile forces [12]. The implant screws are entirely rough, up to the interface fixture-abutment, with OsseoGRIP, characterized by a micro sandblasting through HA of medical degree (HA removed from the implant surface after treatment and superficial decontamination process) for a better osteoconductivity of the titanium oxide surface, validated by 25 years of use in dental implantology. In the surface modification process an important role has been given to the surface decontamination obtained through several washing steps and cold argon plasma treatment, both procedures have been validated and certified [5][6][7][8][9].

#### Features of the implant-prosthetic connection

With the GTB implant we wanted to guarantee the highest mechanical precision in the fixture-abutment coupling allowing the prosthetic components very easy-to-use. A vertical connection with tapered coupling has been chosen to ensure the maximum stability of the abutment, the absence of microgap at the fixture-abutment interface and a coupling that would lead to a cold welding between the tapered coupling interfaces of the implant and the abutment, thus ensuring the absence of micromotion between the two components. The seal between implant and prosthetic abutment is obtained by a conical coupling of 11° each side [10][13][14]. This choice guarantees a proper seal between the fixture and the abutment and ensures an easy positioning/removal of the prosthetic components thanks to 10mm<sup>2</sup> of coupling tapered surface. Further more, a taper with 11° each side allows to optimize the load transfer from the abutment to the implant and consequently to the bone. The fixture-abutment coupling has an antirotational octagonal index ensuring to quickly find the right positioning of the prosthetic components. There is one single diameter of implant-prosthetic connection for any diameter and variation of the implant screws. This choice greatly decreases the number of prosthetic components needed, reduces the costs for the practice and makes easier to transfer information from the practice to the laboratory and viceversa. This type of vertical connection and the presence of a unique diameter of implant-prosthetic connection lead necessarily to an horizontal prosthetic platform shifting: the difference between abutment diameter and implant maximum diameter is increasing according to implant diameter. The GTB implant optimizes the concept of platform shifting, introducing also the vertical platform shifting (for 3.6 - 4.3 - 5.0 implant's neck). The combination of horizontal and vertical platform shifting dramatically increases the coronal osteointegrable implant surface (just think that for 4.3 implant's neck, the combination of horizontal and verti-

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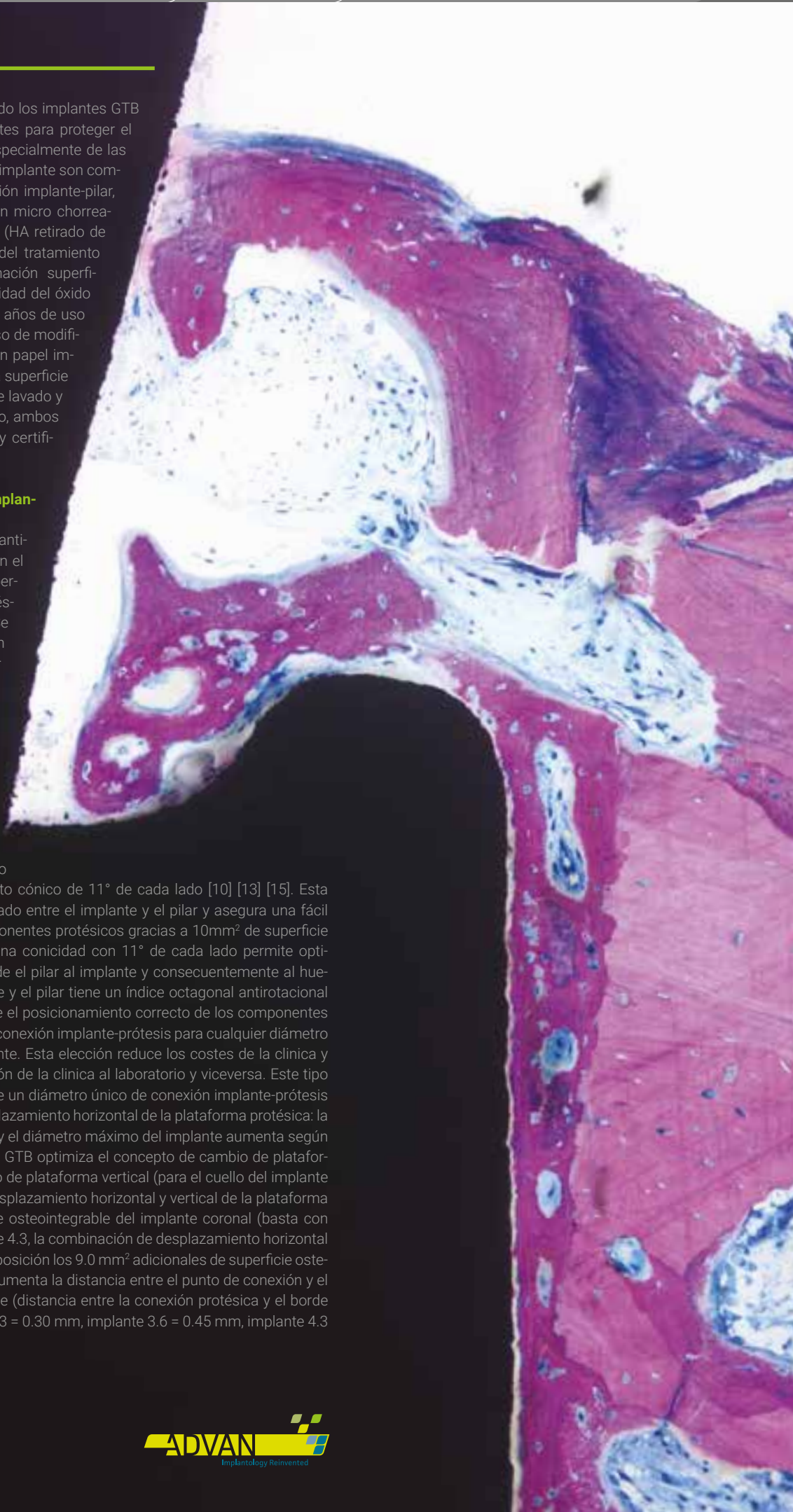
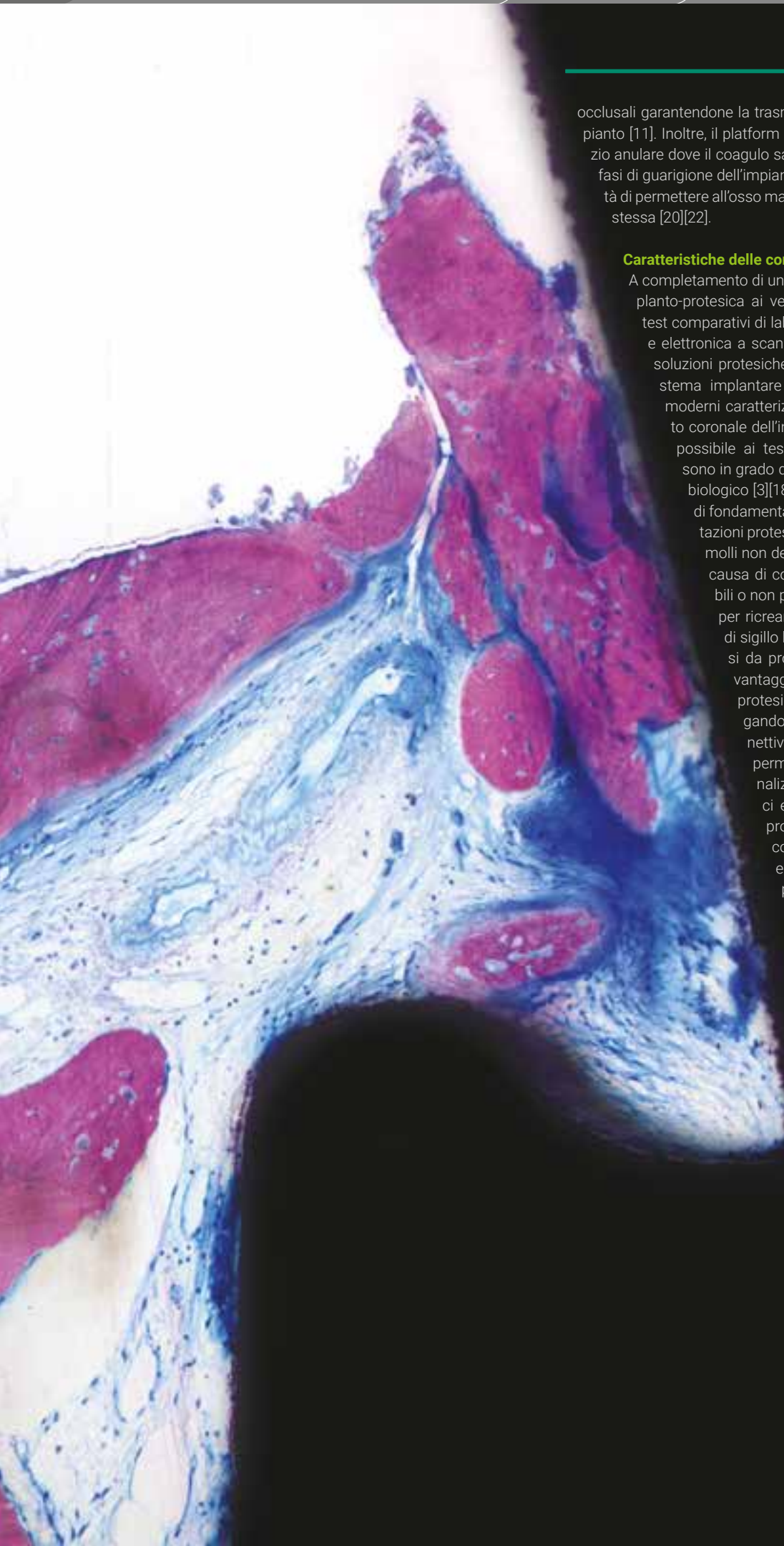
cal platform shifting makes available the additional 9.0 mm<sup>2</sup> of osteointegrable implant surface). Besides, it further distances the fixture-abutment junction point from the outer edge of the implant neck (distance between prosthetic connection and edge of the implant neck 3.3 implant = 0.30 mm, 3.6 implant = 0.45 mm, 4.3 implant = 0.85 mm and 5.0 implant = 1.20mm). Finally a geometrically complex platform shifting like the one of the GTB implant, lowers the point where the load is transmitted through the implant neck, making this implant biomechanically superior even in case of juxta-crestal positioning for the centering of the occlusal forces, ensuring its transmission with the apex of the implant [11]. In addition, the platform shifting of the GTB implant creates an annular space where the blood clot can remain protected during the healing phases of the implant, dramatically improving the ability of the marginal bone to regrow over the platform itself [20][22].

### Features of the prosthetic components

To design the implant screw and the implant-prosthetic connection for superior biomechanical performance using comparative tests of laboratory and optical and electron SEM microscopy, it is then necessary to offer outstanding prosthetic solutions. The prefinished components of the GTB implant system present modern emerging profiles characterized by slim abutments at the implant coronal level ensuring the maximum possible volume to the perimplant soft tissues for a total biological sealing [3] [18]. The protection of the perimplant soft tissue is a key point if aesthetic rehabilitation is required to be stable over time. The soft tissues should not be chronically inflamed due to unstable or unfitted implant-prosthetic connections. The tissues must have enough volume to recreate an appropriate thickness for the biological seal and must not be compressed by inadequate emerging profiles: an advantage for the aesthetic prosthetic rehabilitation. Better aesthetics is also achieved by using prefinished abutments with anatomic profiles to allow it to be used quickly without expensive customization in terms of time and cost. Obviously, The prosthetic components that have their healing bundles differentiated in the maximum emerging diameters in the case of immediate post-extractive implants and have innovative prosthetic solutions that maximize the modern concepts of One Time Abutment [14]. The innovative prosthetic systems of the GTB system, such as the EASy abutments or the GFA multiprotésico system, allow you to run the One Time Abutment approach in all clinical conditions with remarkable operational benefits and simplifications for the dental laboratory. This allows the soft tissue to seal, important to avoid bacterial colonization of the mucosal channel [23], and to optimize the integration of soft peri-implant tissues in order to protect the marginal bone immediately below. Even now, the subcrestal positioning turns out to be an advantageous indication because it allows to increase the width between the prosthetic interface of the implant screw and the coronal mucosal margin, thus guaranteeing compliance with the biological width [16].

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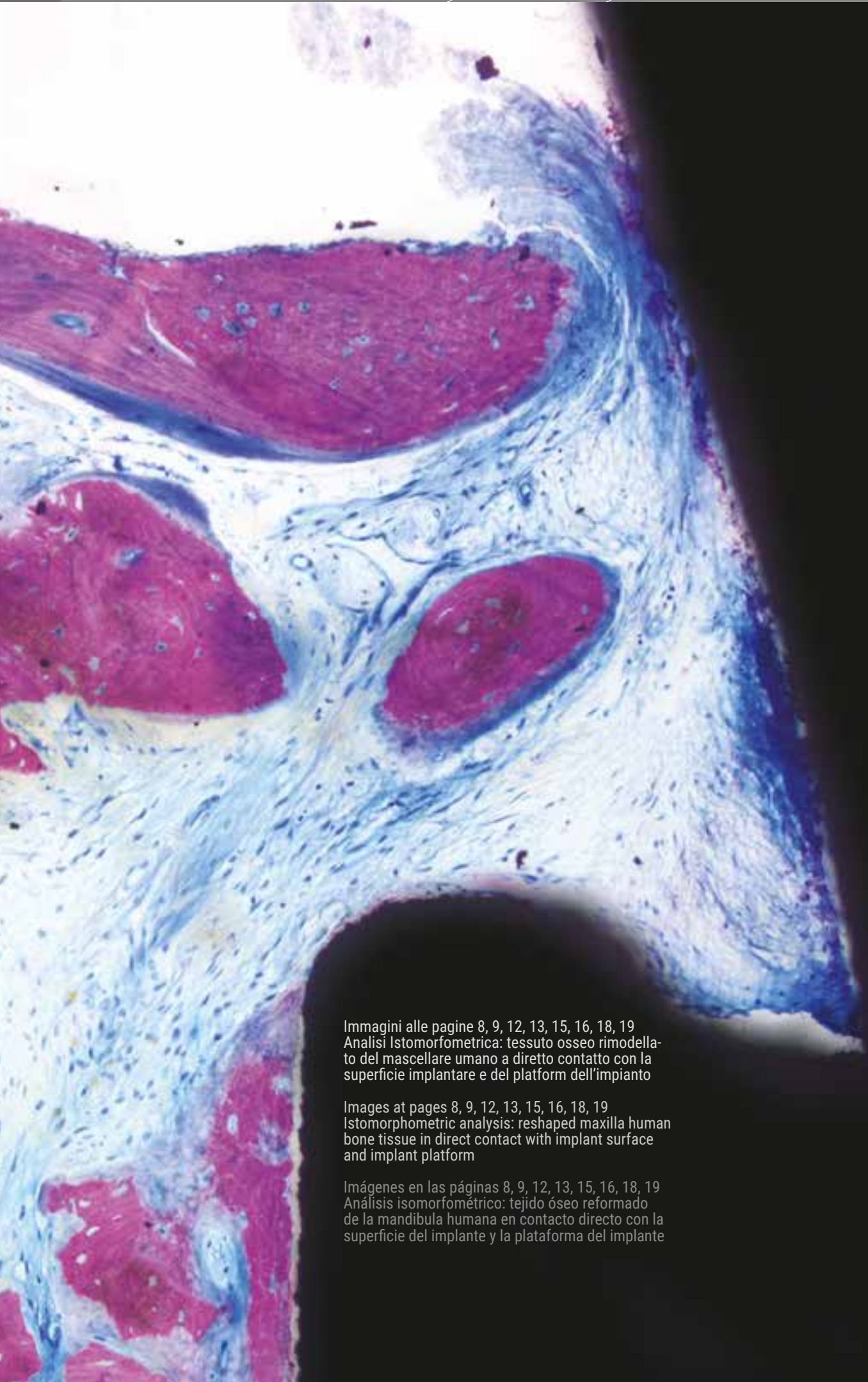
**GTB TUTELA I TESSUTI MOLLI PERI-IMPLANTARI**

**GTB PROTECTS PERI-IMPLANT SOFT TISSUE**

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
Images at pages 8, 9, 12, 13, 15, 16, 18, 19  
Istomorphometric analysis: reshaped maxilla human bone tissue in direct contact with implant surface and implant platform

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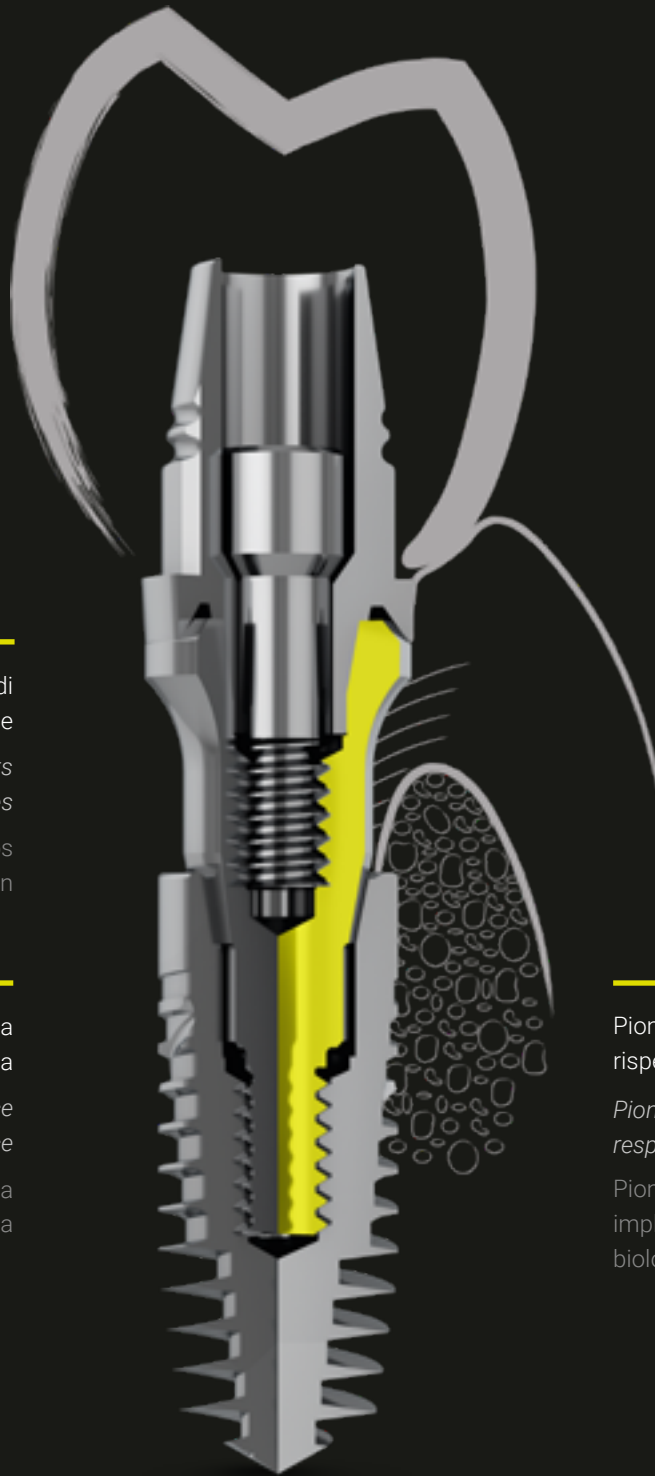*Competence  
Know-How and Scientific Evidence*

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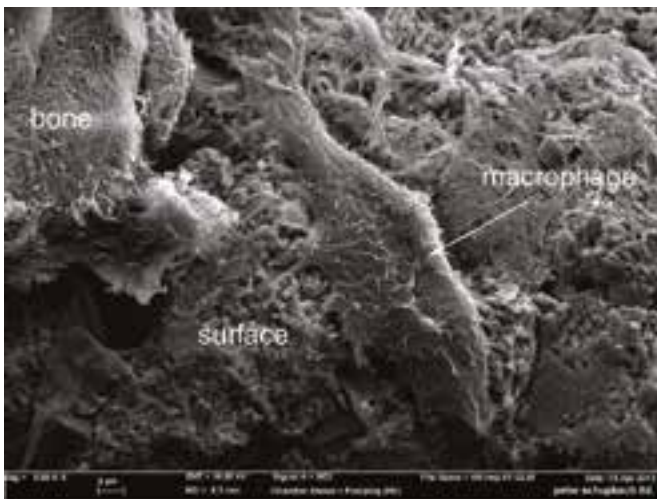
*Pioneers in implant solutions that  
respect human biology*

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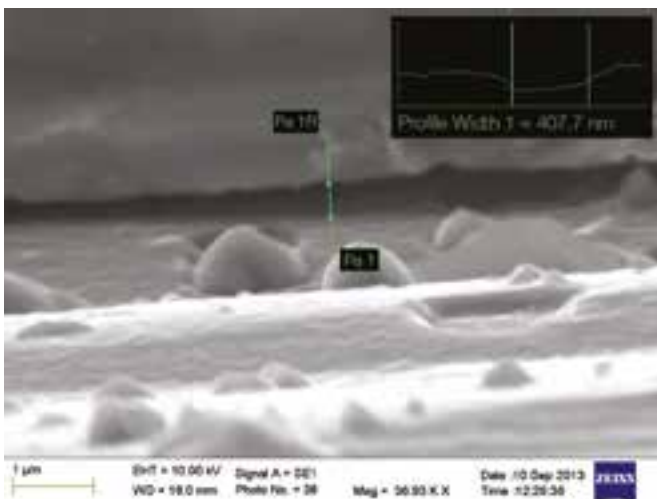
*SEM analysis: fixture-abutment micro-gap*

*Análisis SEM: micro-gap a la union implante-pilar*

- (1) Baggi L, Di Girolamo M, Mirisola C, Calcaterra R: Implant Dent. 2013 Aug;22(4):344-50. doi: 10.1097/ID.0b013e3182943062. Microbiological evaluation of bacterial and mycotic seal in implant systems with different implant-abutment interfaces and closing torque values.
- (2) Baggi L, Cappelloni I, Di Girolamo M, Maceri F, Vairo G: J Prosthet Dent. 2008 Dec;100(6):422-31. doi: 10.1016/S0022-3913(08)60259-0. The influence of implant diameter and length on stress distribution of osseointegrated implants related to crestal bone geometry: a three-dimensional finite element analysis.
- (3) Rompen et al.: J Prosthet Dent. 2007 Jun;97(6 Suppl):S119-25. Soft tissue stability at the facial aspect of gingivally converging abutments in the esthetic zone: a pilot clinical study
- (4) Guo X E 2001. Biomechanics and Remodeling of Trabecular Bone, in Recent Advances in Biomechanics, Edited by C. Zhu and M. Long, Chinese Higher Education Press and Springer-Verlag, pp 120-129
- (5) Biasotto M, Di Lenarda R, Cadenaro M: Effects on titanium surface topography of different surface treatments; J Dent. Res. 2000, 79 (special issue), abstr.1757
- (6) Sandrucci M A, Basa M, Biasotto M, Bareggi R: Fibronectin (FN) arrangement and collagen expression in human fibroblast cultured on titanium surface; Italian Journal of anatomy and embryology (Liv congress società di anatomia, Firenze 21-14 settembre 2000), 2000 vol.105, suppl.1:209
- (7) Di Lenarda R, Biasotto M, De Stefano Dorigo E: Il titanio e le sue leghe in odontoiatria; Odontoiatria 2001, 2:123-130
- (8) Biasotto M, Scuor N, Chiesa R, Schimid C, Di Lenarda R, Cigada A: Caratterizzazione di superfici di titanio per impianti; Biomateriali 2001, 15(1):5-13
- (9) Biasotto M, Cadenaro M, Scuor N, Di Lenarda R, De Stefano Dorigo E: Valutazione comparativa del trattamento di superficie in 5 sistemi implantari; Quintessence International 2002
- (10) Hansson S: Implant-abutment interface: biomechanical study of flat top versus conical. Clin Impl Dent Rel Res 2000;2(1):33-41. ID No. 75159
- (11) Hansson S: A conical implant-abutment interface at the level of the marginal bone impro-

ves the distribution of stresses in the supporting bone. An axisymmetric finite element analysis. *Clin Oral Implants Res* 2003;14(3):286-93. ID No. 79030

(12) Baggi L, Pastore S, Di Girolamo M, Vairo G:  
*J Prosthet Dent.* 2013 Jan;109(1):9-21. doi: 10.1016/S0022-3913(13)60004-9. Implant-bone load transfer mechanisms in complete-arch prostheses supported by four implants: a three-dimensional finite element approach.

(13) Meleo D, Baggi L, Di Girolamo M, Di Carlo F, Pecci R, Bedini R:  
*Ann Ist Super Sanita.* 2012;48(1):53-8. DOI: 10.4415/ANN\_12\_01\_09. Fixture-abutment connection surface and micro-gap measurements by 3D micro-tomographic technique analysis.

(14) Calcaterra R, Di Girolamo M, Mirisola C, Baggi L:  
*Int J Periodontics Restorative Dent.* 2016 Mar-Apr;36(2):275-80. doi:10.11607/prd.2082. Effects of Repeated Screw Tightening on Implant Abutment Interfaces in Terms of Bacterial and Yeast Leakage in Vitro: One-Time Abutment Versus the Multiscrewing Technique.

(15) Giuseppe Vairo, Simone Pastore, Michele Di Girolamo and Luigi Baggi:  
*Implant Dentistry - A Rapidly Evolving Practice Stress Distribution on Edentulous Mandible and Maxilla Rehabilitated by Full-Arch Techniques: A Comparative 3D Finite-Element Approach*  
Edited by Ilser Turkyilmaz, ISBN 978-953-307-658-4, 544 pages,  
Publisher: InTech, Chapters published August 29, 2011 under CC BY-NC-SA 3.0 license DOI: 10.5772/706

(16) Baggi L, Di Girolamo M, Vairo G, Sannino G:  
*Comput Math Methods Med.* 2013;2013:250929. doi: 10.1155/2013/250929. Epub 2013 Jun 19. Comparative evaluation of osseointegrated dental implants based on platform-switching concept: influence of diameter, length, thread shape, and in-bone positioning depth on stress-based performance.

(17) Calcaterra R, Di Girolamo M, Mirisola C, Baggi L:  
*Implant Dent.* 2016 Jun;25(3):348-52. doi: 10.1097/ID.0000000000000379. Expression of Pattern Recognition Receptors in Epithelial Cells Around Clinically Healthy Implants and Healthy Teeth.

(18) L Baggi, R Calcaterra, M Di Girolamo:  
*The pontic crown in the modern implant prosthesis* *DM* 6/2016: 72-74

(19) Rocci A, Calcaterra R, Di Girolamo M, Rocci M, Rocci C, Baggi L:  
*Oral Implantol (Rome).* 2016 Jul 23;8(4):87-95. doi: 10.11138/orl/2015.8.4.087. eCollection 2015 Oct-Dec. The influence of micro and macro-geometry in

term of bone-implant interface in two implant systems: an histomorphometrical study.

(20) Rocci A, Calcaterra R, Rocci M, Rocci C, Di Girolamo M, Baggi L:  
*Oral Implantol (Rome).* 2016 Nov 13;9(1):11-16. doi: 10.11138/orl/2016.9.1.011. eCollection 2016 Jan-Mar. Different performance of platform switching in equicrestal position implant: an histological study.

(21) Small & Tarnow:  
*Gingival recession around implants: a year longitudinal prospective study.* *JOMI*, 2000;15(4):527-532

(22) Iezzi G, Iaculli F, Calcaterra R, Piattelli A, Di Girolamo M, Baggi L:  
*Oral Implantol.* 2017 Jun;43(3):180-186. doi: 10.1563/aaid-joi-D-16-00182. Epub 2017 Mar 16. Histological and Histomorphometrical Analysis on a Loaded Implant With Platform-Switching and Conical Connection: A Case Report.

(23) Meynardi F, Pasqualini ME, Rossi F, Dal Carlo L, Nardone M, Baggi L:  
*Oral Implantol (Rome).* 2017 Feb 14;9(Suppl 1/2016 to N 4/2016):10-20. doi:10.11138/orl/2016.9.1S.010. eCollection 2016 Jan-Mar. Implant Dentistry: Monitoring of Bacteria Along the Transmucosal Passage of the Healing Screw in Absence of Functional Load.

(24) Di Girolamo M, Arullani CA, Calcaterra R, Manzi J, Arcuri C, Baggi L:  
*Oral Implantol (Rome).* 2016 Nov 16;9(4):222-232. doi: 10.11138/orl/2016.9.4.222. eCollection 2016 Oct-Dec. Preservation of extraction socket in immediate implant placement: a clinical study.

(25) Di Girolamo M, Calcaterra R, Di Gianfilippo R, Arcuri C, Baggi L:  
*Oral Implantol (Rome).* 2016 Nov 13;9(1):1-10. doi: 10.11138/orl/2016.9.1.001. eCollection 2016 Jan-Mar. Bone level changes around platform switching and platform matching implants: a systematic review with meta-analysis.

(26) M. Di Girolamo, L. Baggi, P. Pirelli, L. Pappalardo, G. Massei, F. Iaculli, G. Iezzi, A. Piattelli, R. Calcaterra:  
*Oral Implantol (Rome) - Anno X - N. 3/2017*  
3DX-Ray microscopic analysis on a prosthetically loaded implant with platform -switching and conical connection: a case report.

(27) Qing-Qing Wang, Ruoxi Dai, Chris Ying Cao, Hui Fang, Min Han, Quan-Li Li:  
*Plos One* October 19, 2017  
One-time versus repeated abutment connection for platform-switched implant: a systematic review and meta-analysis.

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**The GTB is an osteointegrated dental implant system with controlled and gradual transmission of the masticatory load**

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Cylindrical profile of the implant neck  
*Minimizes the stress of the cortical bone*

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Double, progressive and modified reverse buttress thread  
*Faster insertion, better primary stability, ideal load transmission. Enhanced BIC*

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Conical profile of the implant root  
*Ideal distribution of the masticatory load, minimal tensile and maximum compressive forces on the bone*

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**Geometrically complex Switch Platform, high precision mechanical coupling, easy to use prosthetic components**

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Full surface treatment OsseoGRIP  
Maximization of BIC with marginal remodeled bone

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Fluids discharge channel  
Reduction of the hydrostatic compression of the bone clot

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Horizontal and vertical Platform Shifting: BioPlatform  
Allows bone growth over the platform, increases space for the periimplant soft tissues, reduces the stress to implant neck lowering the application point of the forces generated by load

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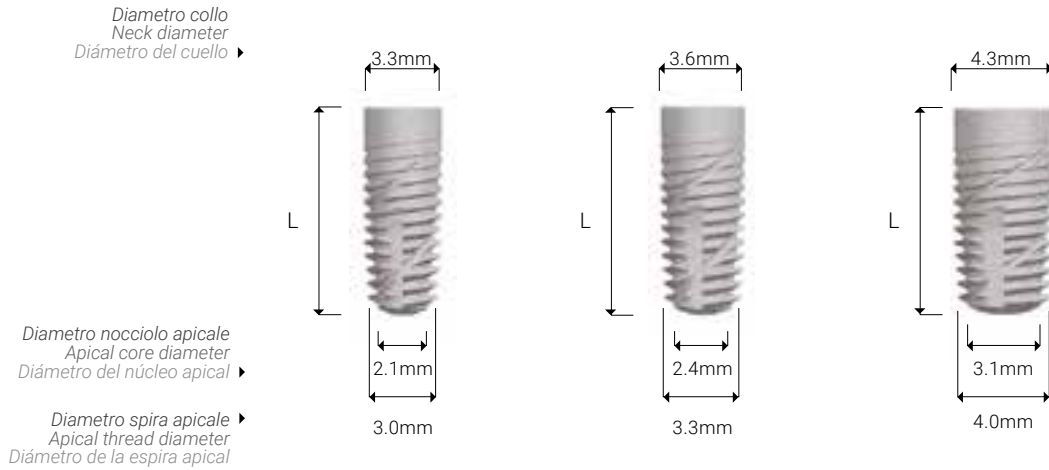
Apical anti-rotation notch  
Improves self-tapping performance of the implant, improves primary stability

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The implant is packaged without pre-assembled mounting device. The standard cover screw is included in the package. The self-centering design of cover screw allows easy insertion and tightening. Use prosthetic driver to tighten the cover screw.

**For more information on the implant, packaging and surgical procedures refer to the GTB Surgical Manual.**

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**The GTB-Tzero is an osteointegrated dental implant system with controlled and gradual transmission of the masticatory load specifically designed for immediate load**

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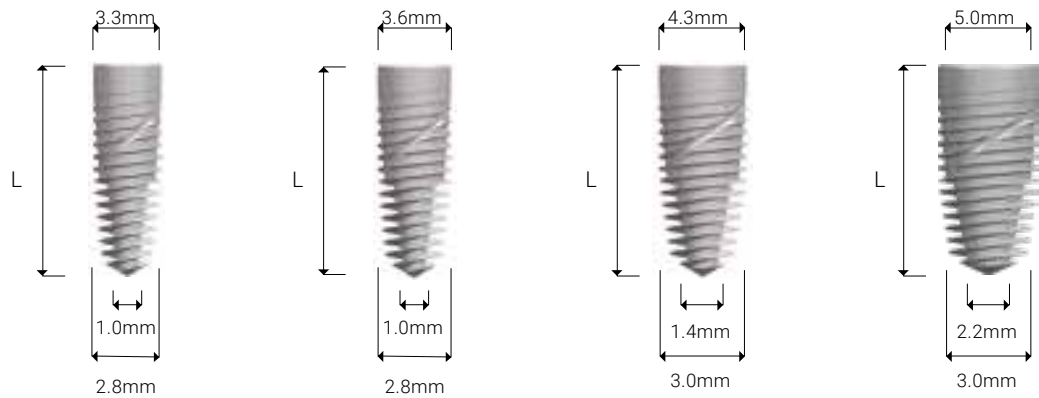
The implant is packaged without pre-assembled mounting device. The standard cover screw is NOT included in the package. The self-centering design of cover screw allows easy insertion and tightening. Use prosthetic driver to tighten the cover screw.

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The images shown are for illustrative purposes only.

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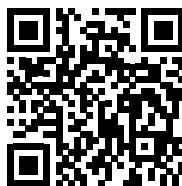
The instructions for use are available at the following Internet address or by scanning the QR code.

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**IMPLANT SCREW PACKAGING**

The implant screws are processed, cleaned and treated accordingly to certified procedures. The final decontamination is performed by cold plasma of argon and the implants are immediately stored. The implant screws are removed from storage only inside the Clean Room of the production factory. In this way, the superficial oxide layer will be formed in a controlled atmosphere, not allowing the slightest contamination of the implant surface. After packaging in the Clean Room, the implant screws are sent to the sterilization process by  $\beta$  rays.

The packaging is characterized by:

- Double sterility: outer barrier provided by blister PET-G with sheet in TYVEK®; internal barrier provided by the vial in polycarbonate
- Cover screw positioned in the vial cap under a seal in TYVEK®
- Shaped blister to avoid impacts to the vial
- Implant screws in contact only with elements in titanium, to avoid phenomena of bimetalism or contamination from contact with plastic material

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### SURGICAL KIT PRO

The surgical kit consists of a single box containing all the surgical and prosthetic instruments necessary for the realization of the osteotomy, the implant positioning and for its prosthesis. Including Radel made surgical tray (non-toxic, autoclavable, metal-free). **OPTIONAL: available the set of drills for TZERO EXTRA implant diameter 5.0 mm ref. TDS50.**

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**DRILLS STOP KIT**

The stop kit consists of a single mini-box containing all the stops used with bone drills series. Including tray Radel made (non-toxic, autoclavable, metal-free). OPTIONAL KIT

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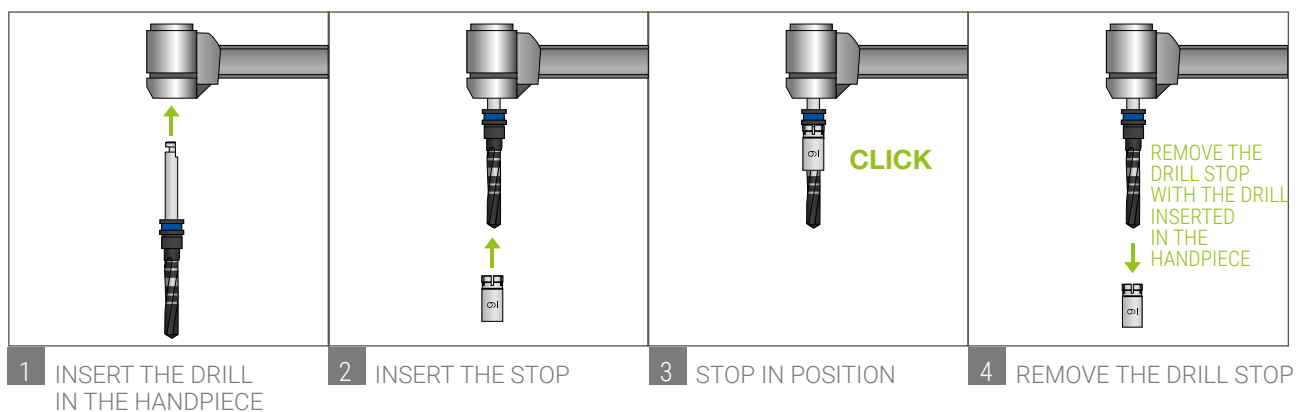
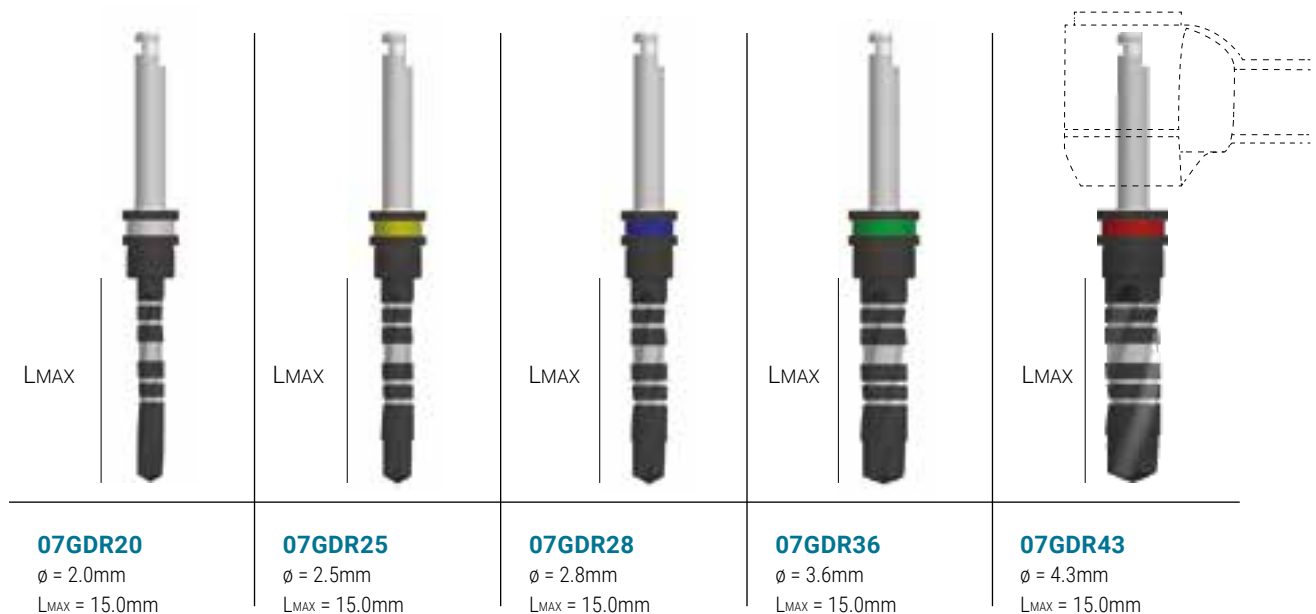
### ALVEOLAR BONE DRILLS

The alveolar bone drills have a single ring color code to easily recognize the diameter. The alveolar bone drills have a smaller apex diameter to create the correct implant preparation with a soft and cancellous bone. Milled and laser marked work depth references. The drills produce an osteotomy that can be considered congruent to the length of the implant chosen as the maximum over preparation depth it's just 0.4 mm (due to the conical apex of the drill unlike the convex apex of the implant). External irrigation drills. Diamond Like Carbon Surface Finish.

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### COMPACT BONE DRILLS

The compact bone drills have a double ring color code to easily recognize the diameter. The compact bone drills have a cylindrical profile to create the correct implant preparation with a hard and compact bone. Milled and laser marked work depth references. The drills produce an osteotomy that can be considered congruent to the length of the implant chosen as the maximum over preparation depth it's just 0.4 mm (due to the conical apex of the drill unlike the convex apex of the implant). External irrigation drills. Diamond Like Carbon Surface Finish.

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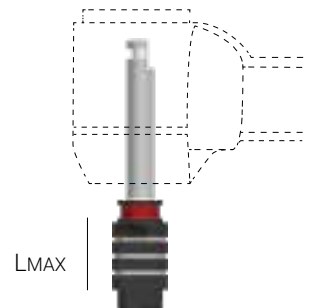
### MARGINAL BONE DRILLS

The marginal bone drills have a single ring color code to easily recognize the diameter. The marginal bone drills have a self-centering apex. Milled and laser marking work depth references. External irrigation drills. Diamond Like Carbon Surface Finish.

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### OPTIONAL DRILLS

Optional drills allow to mark the bone for its preparation (Marking Burr) and easily drill the cortical bone (Pointed Starter Drill). The Lindemann Drill allows to correct the axis of the implant site after using the first surgical bone drill. The Drill Extension allows to increase the length of the handpiece tools.

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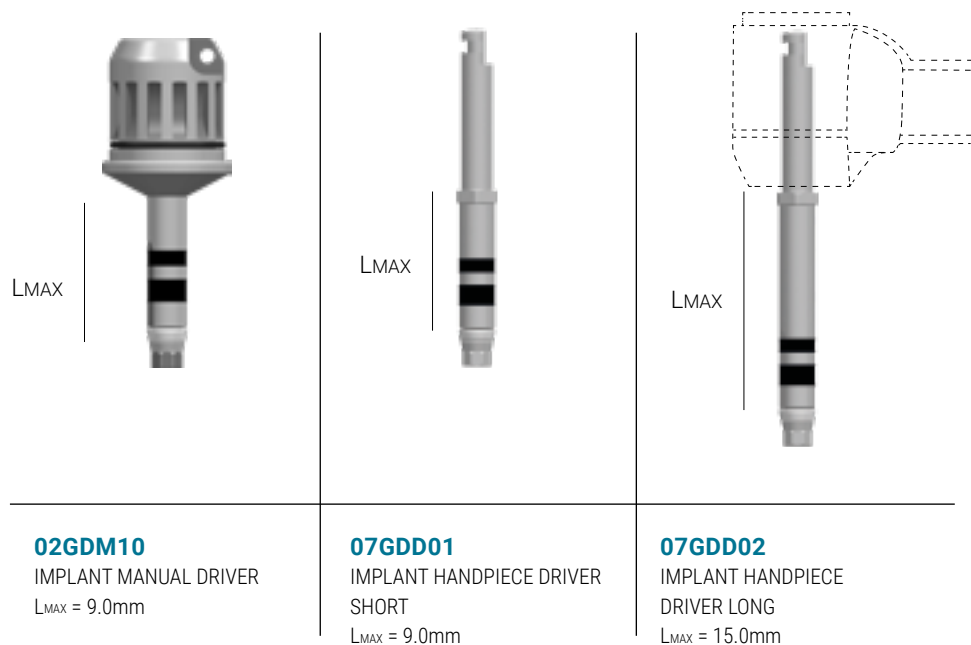
## IMPLANT DRIVERS

Implant drivers have a retention system to engage the implant, pick it up, carry it, and finalize positioning. Implant drivers feature the W&H hexagonal system. Implant drivers have laser marked references that indicate the transmucosal heights of pre-defined prosthetic abutments (positioned at 2.0-3.5-4.5-5.5 mm). They also have a reference that allows to control radial position of the implant's octagonal index (indicating the position of the vertex of the octagon).

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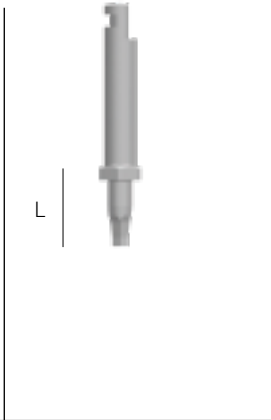
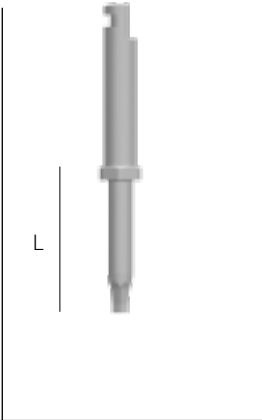
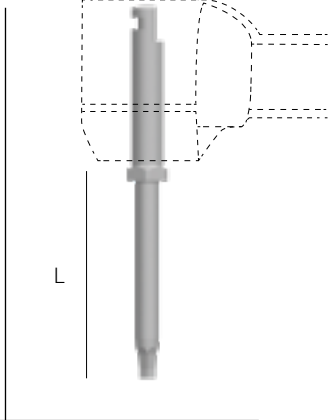
### PROSTHETIC DRIVER

The GTB implant system has only one type of prosthetic driver (hexagon 1,2 mm) to screw and unscrew prosthetic components such as surgical screws, healing abutments, retaining screws for prosthetic components and transfer abutments. The prosthetic drivers feature the W&H hexagonal system.

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				
<b>02EGM05</b> PROSTHETIC MANUAL DRIVER SHORT L = 3.0mm	<b>02EGM10</b> PROSTHETIC MANUAL DRIVER L = 10.0mm	<b>07-EG05</b> PROSTHETIC HANDPIECE DRIVER SHORT L = 5.0mm	<b>07-EG10</b> PROSTHETIC HANDPIECE DRIVER MEDIUM L = 10.0mm	<b>07-EG15</b> PROSTHETIC HANDPIECE DRIVER LONG L = 15.0mm

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### DYNAMIC DRIVER

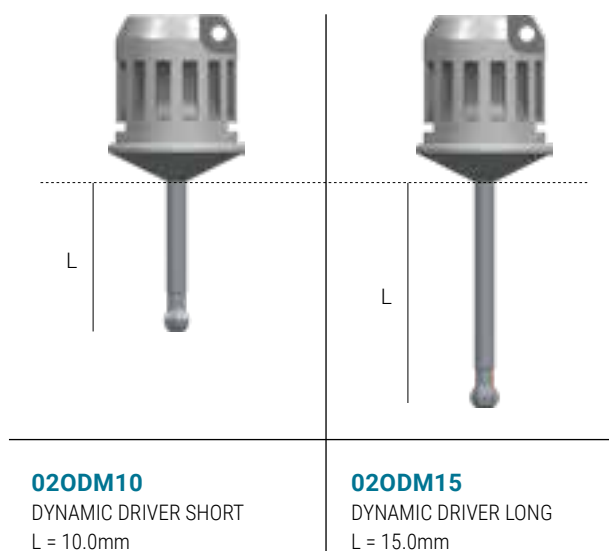
The GTB system has a specific prosthetic driver for screwing angled respect to the implant axis. Useful solution for screwed prosthetic rehabilitations, this dynamic driver allows the driver to be angulated up to 29° respect to the implant axis.

PLEASE NOTE: the dynamic driver is not compatible with standard retaining screws and must only be used with the the dynamic screws.

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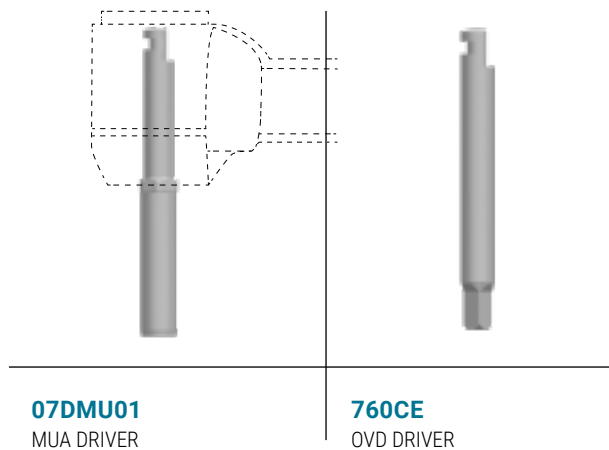
### MUA AND OVERDENTURE DRIVER

Specific drivers for straight MUA abutments and overdenture abutments. The MUA drivers feature the W&H hexagonal system.

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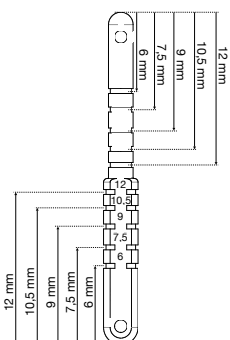
**ACCESSORIES**

The GTB system features accessories to make easy specific surgical or prosthetic operations. Parallel pins are graduated and therefore can also be used to measure the osteotomy depth. The depth probe is graduated and has an apical cling to verify the position of the basal cortex during maxilla great sinus lift. The torque wrench has an adjustable screw-in torque control (10-70 Ncm) and can be used both in the surgical and prosthetic phase. The ratchet adapter allows to manually use all handpiece tools and can be used with the torque wrench. Titanium forceps allow to handle the implant without any contamination if needed. The surgical driver allows to position an implant or screw a retaining screw out of occlusion.

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### BONE BOWL

The surgical kit can be completed with a bone bowl useful to correctly manage autologous bone or bone graft.

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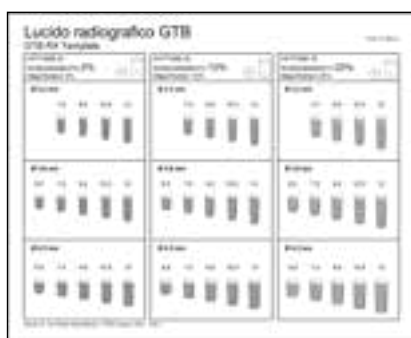
### IMPLANT SIZING OVERLAY

The implant-sizing overlay allows the correct planning with TC, RX ad OPT.

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For more information on the implant, packaging, instruments and surgical procedures refer to GTB Surgical Manual.

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The BORN DIGITAL logo indicates prosthetic components suitable for the digital environment

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### Notice

Il logo ATiM indica componenti adatte ai protocolli One Time Abutment

ATiM logo indicates components suitable for One Time Abutment protocols

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**Modern prosthetic solutions for durable aesthetics.  
Unique prosthetic connection for every fixture's diameter and type**

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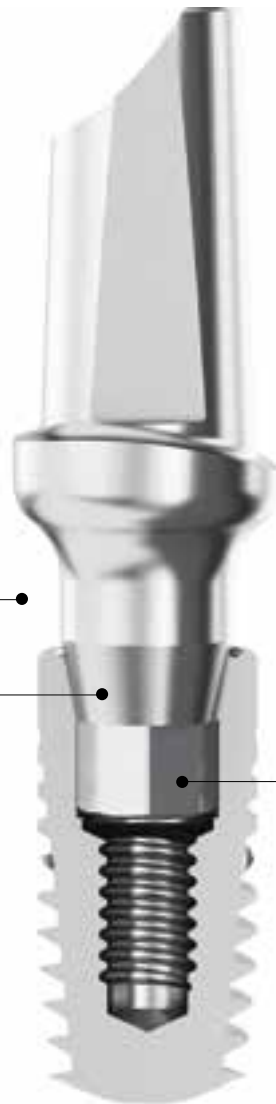
Unique prosthetic platform  
and different transmucosal profiles  
different by height  
*Possibility to always choose the most suitable  
abutment for the element to be restored.  
Simplifies management of prosthetic  
components and reducing codes number*

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Coupling cone angle 11° (each side),  
1.2 mm height  
*Maximum seal between implant and abutment,  
absence of bacterial infiltration, easily removable  
prosthetic components, retention screws  
tightening torque reduction of the prosthetic  
components*

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Slim and concave profile of the abutment  
*Maximization of the available space  
for the peri-implant soft tissue, profiles suitable  
for the subcrestal implant positioning*

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Antirotational octagonal index  
1.4 mm height  
*Ideal trade off between easy to find correct  
position and number of possible positions.  
Possibility of using the pre-angled  
components, due to the presence of a vertex every 45°,  
safety during the positioning of the prosthetic  
components due to the connection depth*

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### COVER SCREW

Surgical cover screws available in three different heights. Standard height cover screw (already included in the implant vial, excluding the GTB-TZERO implant) suitable for sub-crestal positioning of the implant. Zero-height cover screw for juxta-crestal positioning of the implant and reduced gingival thickness. Increased height cover screw suitable for deep sub-crestal positioning or as support for membranes in case of bone regeneration. Sterile packaging. Made in Ti gr.23.

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**HEALING ABUTMENTS**

They allow proper mucosal channel conditioning. The height of the healing abutments is slightly higher than the corresponding prosthetic abutments so that the surrounding mucosa is slightly overcome. All profiles are enlarged in the emerging diameter respect the corresponding prosthetic abutments to allow an easy positioning, comfortable for the patient without use of anesthetic. Also available in diameter LARGE and X-LARGE for immediate post-extracting implants or to condition the mucosal channel without flaps sliding. Sterile packaging. Made in Ti gr.23.

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4.6mm	6.0mm	7.0mm	<p>H<sub>TOT</sub></p> <p>H CORRESPONDING ABUTMENT HEIGHT</p>
<p><b>04GMG12</b> EASy HEALING ABUTMENT H = 2.0mm H<sub>TOT</sub> = 3.0mm</p> <p><b>04GMG13</b> EASy HEALING ABUTMENT H = 3.5mm H<sub>TOT</sub> = 4.5mm</p> <p><b>04GMG14</b> EASy HEALING ABUTMENT H = 4.5mm H<sub>TOT</sub> = 5.5mm</p> <p><b>04GMG15</b> EASy HEALING ABUTMENT H = 5.5mm H<sub>TOT</sub> = 6.5mm</p>	<p><b>06GMG12</b> LARGE HEALING ABUTMENT H = 2.0mm H<sub>TOT</sub> = 3.0mm</p> <p><b>06GMG13</b> LARGE HEALING ABUTMENT H = 3.5mm H<sub>TOT</sub> = 4.5mm</p> <p><b>06GMG15</b> LARGE HEALING ABUTMENT H = 5.5mm H<sub>TOT</sub> = 6.5mm</p>	<p><b>07GMG12</b> X-LARGE HEALING ABUTMENT H = 2.0mm H<sub>TOT</sub> = 3.0mm</p> <p><b>07GMG13</b> X-LARGE HEALING ABUTMENT H = 3.5mm H<sub>TOT</sub> = 4.5mm</p> <p><b>07GMG15</b> X-LARGE HEALING ABUTMENT H = 5.5mm H<sub>TOT</sub> = 6.5mm</p>	<p>Use Prosthetic Driver Tightening torque 7Ncm</p>

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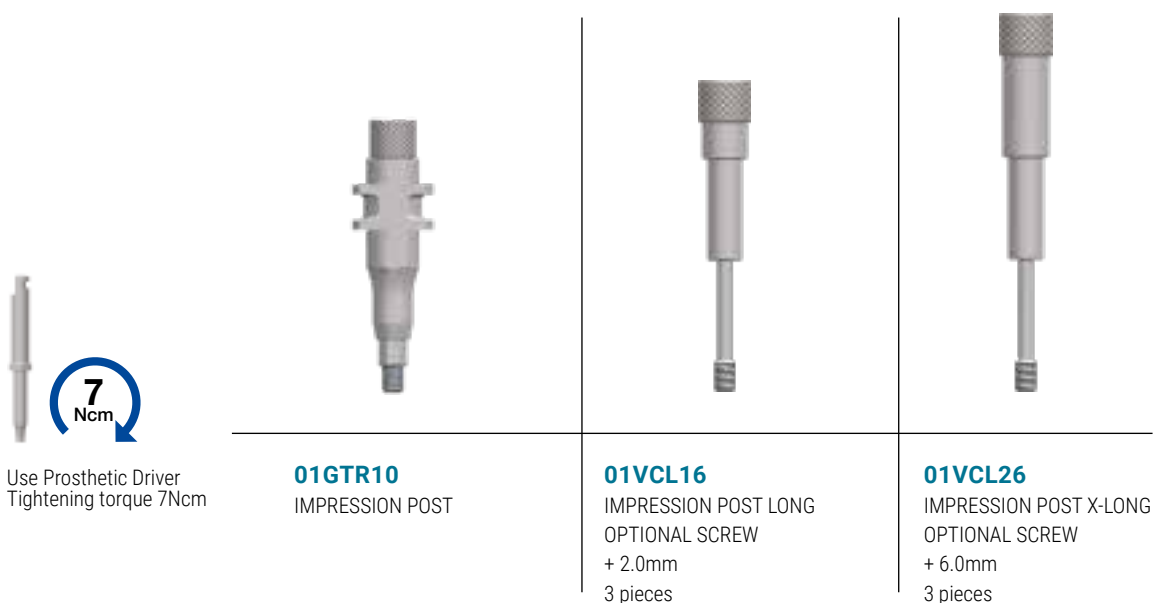
### IMPRESSION POST

Impression post for open tray technique. This impression post allows to remove the octagonal index before removing the impression tray to eliminate any undercuts created by the index in case of multiple non-parallel implants. This allows to quickly remove the impression tray and not to deform the elastomer while removing the tray, obtaining an absolute precision master model. To reposition the octagonal index just re-insert it into the retentive body embedded in the elastomer and screw the impression pin (to properly assemble the removable octagon with the octagon of the retentive body it is advisable to use a screwdriver by inserting it into the notches at the upper end of removable octagonal index). The transmucosal profile is equal to the slimmer mucousal channel that can be obtained with healing abutment to ensure that the impression post can be always easily positioned. Made in Ti gr.23.

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### IMPLANT SCAN ABUTMENT

Abutment for digital impression. Made in Ti gr.23 sandblasted.

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**04GSB10**

IMPLANT SCAN ABUTMENT

Use Prosthetic Driver  
Tightening torque 7Ncm  
Retaining screw TYPE A  
included

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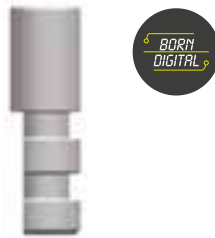
### IMPLANT REPLICA

Implant analog replicating the implant connection. Made in Ti gr.23.

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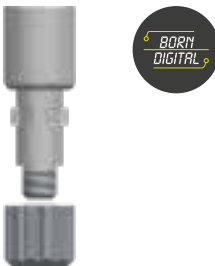
### DIGITAL REPLICA

The digital replica allows to work in the InLab software environment. Standard platform 4.2 mm. Made of Ti gr.23

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**TI-BASE ABUTMENT**

Ti-Base is the titanium base for CAD/CAM rehabilitation. It contributes to low cost workflows for the production of custom abutments.

- Accurate digital acquisition of the implant position through the scan body: extraoral on the model or intraoral.
- Designing abutment with CAD software.
- Realization of abutment mesostructure or crowns from a block.
- Ti-Base adhesive connection with sintered mesostructure or crown abutment.
- Available in three heights to allow maximum flexibility for customized restoration and with respect for biology.
- Compatible with Cerec Standard S by Dentsply Sirona

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Hecho en Ti gr.23.

**BORN DIGITAL**

Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE A  
included

<p><b>41GLC00</b> D = 4.1mm H = 0.7mm</p>	<p><b>41GLC02</b> D = 4.1mm H = 2.0mm</p>	<p><b>41NLC30</b> D = 4.1mm H = 3.5mm</p>	<p><b>41GLC04</b> D = 4.1mm H = 4.5mm</p>
---	---	---	---

**Notice!**  
Esclusivamente per impianto JUXTA-OSSEO.  
Exclusively for BONE LEVEL implant.  
Exclusivamente para implante YUXTA-ÓSEO.

Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE B  
included

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## UNI-BASE ABUTMENT

Uni-Base is the titanium base for CAD/CAM rehabilitation. It contributes to low cost workflows for the production of custom abutments.

- Accurate digital acquisition of the implant position through the scan abutment: extraoral on the model or intraoral.
- Designing abutment with CAD software.
- Realization of abutment mesostructure or crowns with CAM milling machine.
- Adhesive connection between Uni-Base and mesostructure or crown abutment.
- **Sleeve customizable in height to guarantee the prothesist the correct verticality remaining in an integral digital workflow. (UNI-BASE TALL)**

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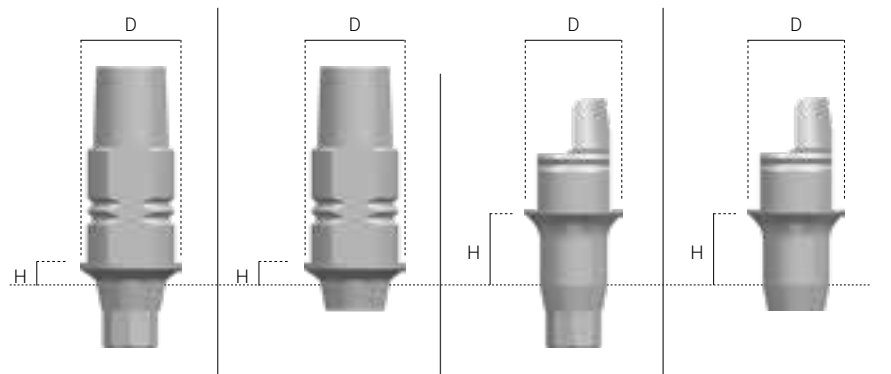
Hecho en Ti gr.23.

Use Prosthetic Driver.  
Tightening torque 25 Ncm.  
Retaining screw TYPE A included.

### 41CUB20

UNI-BASE TALL  
D = 4,1 mm  
H = 0,7 mm  
LOCKING

### 41CUB30

UNI-BASE TALL  
D = 4,1 mm  
H = 0,7 mm  
NOT LOCKING

### 41UNN02

UNI-BASE  
D = 4,1 mm  
H = 2,0 mm  
LOCKING

### 41UNR02

UNI-BASE  
D = 4,1 mm  
H = 2,0 mm  
NOT LOCKING

### 42CUB20

UNI-BASE  
D = 4,1 mm  
H = 3,5 mm  
LOCKING

### 42CUB30

UNI-BASE  
D = 4,1 mm  
H = 3,5 mm  
NOT LOCKING

### 41UNN04

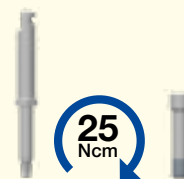
UNI-BASE  
D = 4,1 mm  
H = 4,5 mm  
LOCKING

### 41UNR04

UNI-BASE  
D = 4,1 mm  
H = 4,5 mm  
NOT LOCKING

### Notice!

Esclusivamente per impianto JUXTA-OSSEO.  
Exclusively for BONE LEVEL implant.  
Exclusivamente para implante YUXTA-ÓSEO.

Use Prosthetic Driver.  
Tightening torque 25 Ncm.  
Retaining screw TYPE B included.

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### PRE-MILLED ABUTMENT

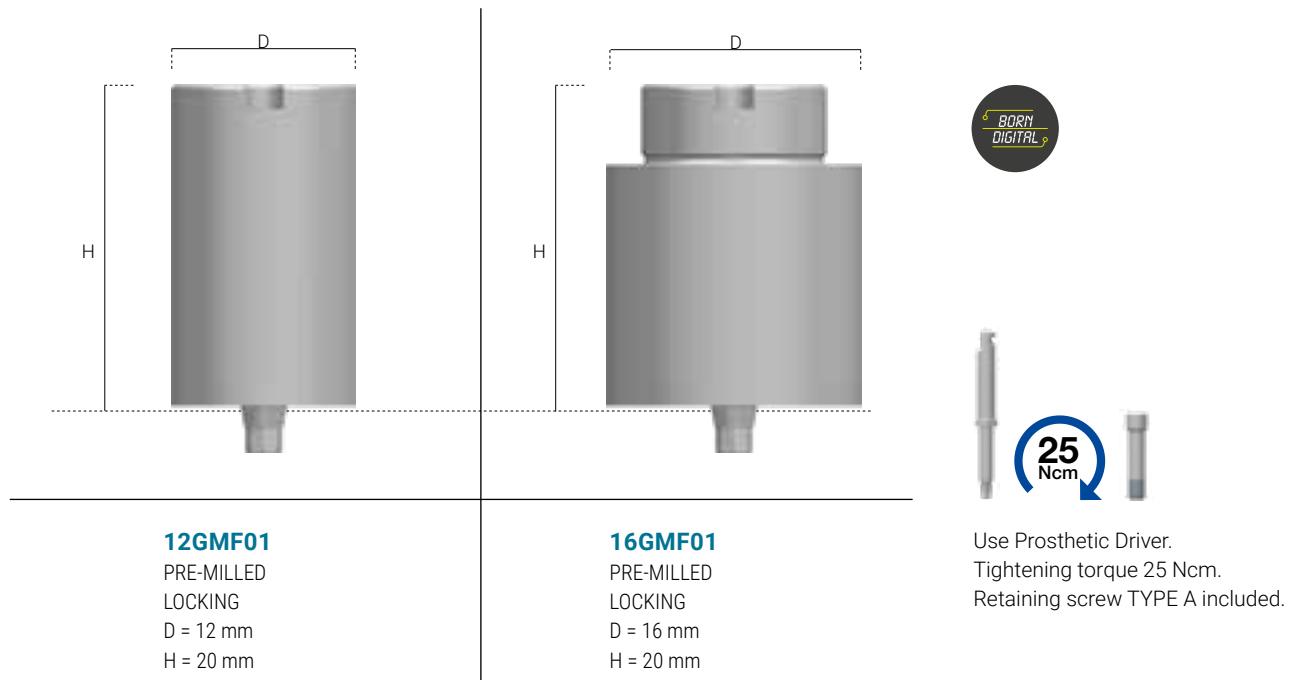
This prosthetic component is equipped with a premilled connection with all the characteristics and all the strict tolerances of implant system guaranteed by Advan. The Pre-Milled abutment allows you to have customized transgingival design, regardless implant position; moreover this prosthetic system reduces production times and allows to orient each insertion axis of the secondary prosthesis.

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### TITANIUM TEMPORARY ABUTMENT

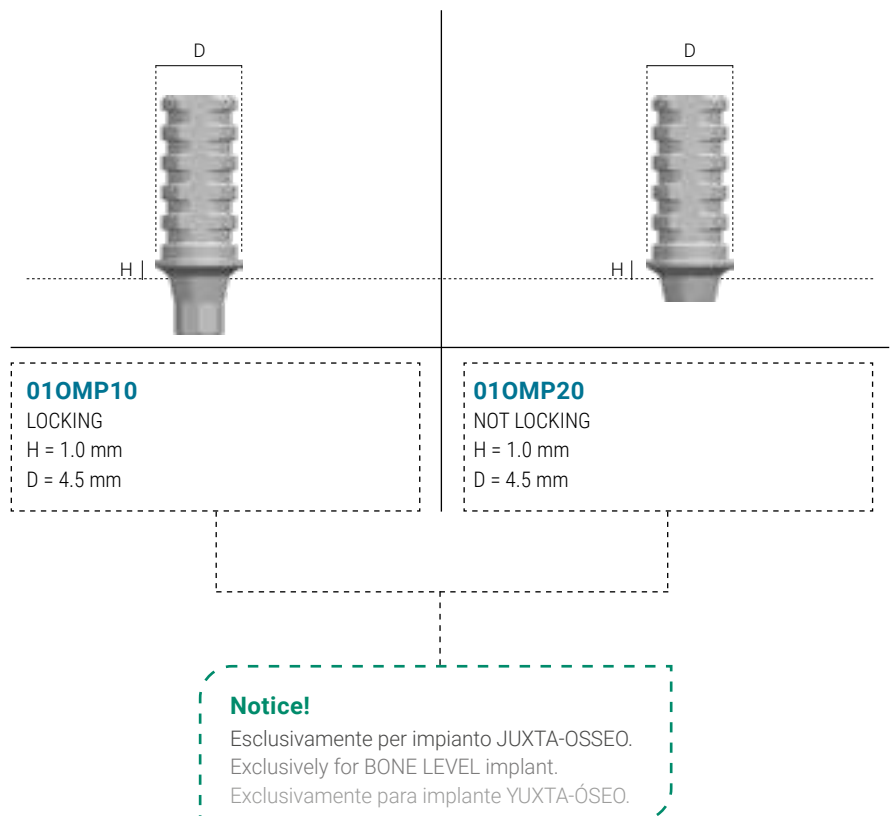
Titanium abutment useful to realise screw-retained temporary prosthodontic rehabilitations. Made of Ti gr.23.

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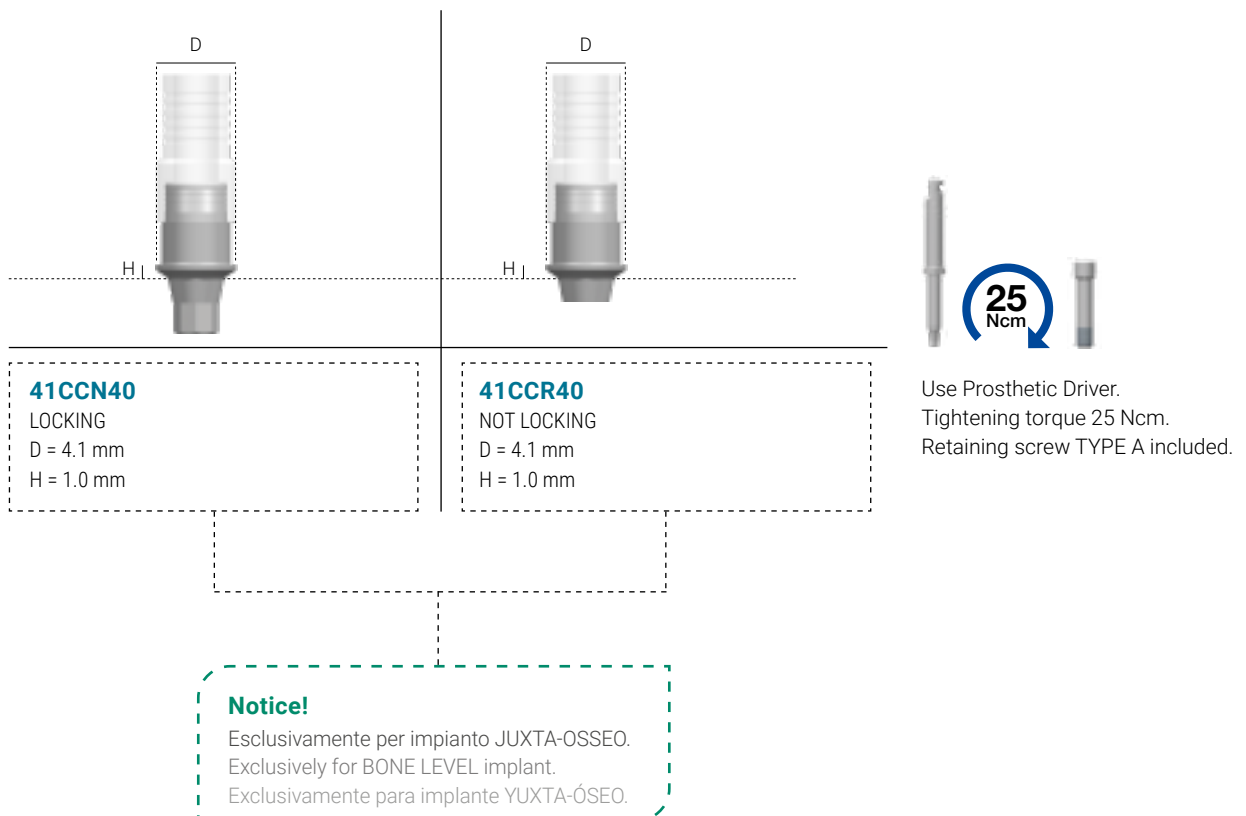
### OVER-CASTING ABUTMENT WITH CoCr28Mo BASE

A castable acrylic abutment with a machined CoCr28Mo base for casting. The abutment is designed for single-crown or multiple-unit bridge restorations and is also commonly used for bar supported overdenture restorations. The precisely machined CoCr28Mo base provides an absolutely precise fit to the implant. The abutments can be milled and individually modeled for a better emergence profile and aesthetically natural-looking result. It is directly connected to the implant through the retaining screw. Castable sleeve made of acetal resin.

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### EASy

The EASy prosthetic system was designed to provide the dentist with a ready to use prosthetic solution, available with a large combination of transmucosal heights and angles, allowing him to quickly perform high-value prosthetic rehabilitation.

The transmucosal profile of the EASy abutment follows the GTB philosophy to easily allow the sub-crestal positioning surgical technique and the correct conditioning of peri-implant tissues in order to comply with the biological width.

The abutment profile is already shaped in the coronal area so that it can be quickly used to reline temporary crowns without having to model the abutment itself.

Finally, the skin-caps, provided with the EASy abutments, allow to take full advantage of the One Time Abutment approach, allowing to make abutment's direct impression with closed-tray technique and quickly reline provisional crowns.

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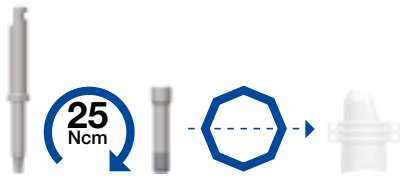
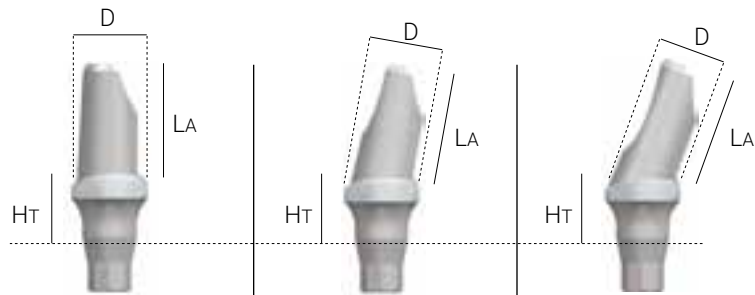
### EASy ABUTMENT

Pre-finished abutment that allows quick high-esthetic prosthetic rehabilitations as well as quick close tray impressions. Thanks to the EASY Skin Cap, these prosthetic components are particularly suitable for immediate loading protocols. The very thin transmucosal profile always guarantees the maximum volume for peri implant soft tissues. The predetermined angulation of the abutment allows to easily correct the disparallelism of the implant. The abutment's pin has an anti-rotational face and an anatomical profile to reduce its size. Made in Ti gr.23.

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Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE A included  
Angled respect octagon vertex  
EASy Skin Cap included (2 pieces)

#### 46GED20

D = 4.0mm  
HT = 2.0mm  
LA = 6.0mm  
 $\angle = 0^\circ$

#### 46GED35

D = 4.0mm  
HT = 3.5mm  
LA = 6.0mm  
 $\angle = 0^\circ$

#### 46GEA210

D = 4.0mm  
HT = 2.0mm  
LA = 6.0mm  
 $\angle = 10^\circ$

#### 46GEA310

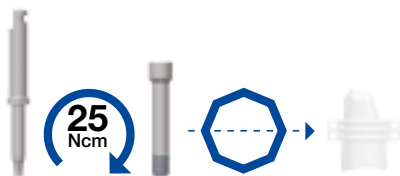
D = 4.0mm  
HT = 3.5mm  
LA = 6.0mm  
 $\angle = 10^\circ$

#### 46GEA220

D = 4.0mm  
HT = 2.0mm  
LA = 6.0mm  
 $\angle = 20^\circ$

#### 46GEA320

D = 4.0mm  
HT = 3.5mm  
LA = 6.0mm  
 $\angle = 20^\circ$

Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE B included  
Angled respect octagon vertex  
EASy Skin Cap included (2 pieces)

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### EASy SKIN CAP & EASy LAB REPLICA SET

These retentive caps allow three fundamental functions during the immediate loading prosthetic rehabilitations with EASy abutments:

- Make a direct impression of EASy abutment
- Reline a temporary crown for EASy abutment
- Realize the cap fusion of the definitive crown for EASy abutment

The EASy Skin Caps feature external retentions both for the impression phase and for the provisional resin relining phase. The retentions can be easily removed with drills or with scalpel to better suit the cap to the specific situation. The EASy Skin Caps internally feature a flat face which matches the one of the easy abutments. The coupling marginal point between the EASy Skin Caps and the EASy abutment is already achieved with precision by the cap and should not be defined with consequent waste of time both in the relining phase of the provisional crown and in the modelling phase of the fusion waxing for the definitive cap. The EASy Skin Caps for EASy abutment present 5 markers that make them suitable as a scan body for EASy abutments. Made of Polycarbonate. To transfer the abutment's position or to quickly reline a temporary crown is available the EASy Laboratory Replica Set, composed of 3 replicas of the EASy abutments (one for each angle) which can be positioned on the implant analogue making them an EASy abutment analogue. Made of Ti gr.23 with blue anodization.

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Position the Skin Cap on the EASY abutment correctly aligning the antirational flat face

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Check the correct positioning on the EASY abutment finishing margin

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Take the impression or relin the temporary crown (if necessary remove the external retentions)

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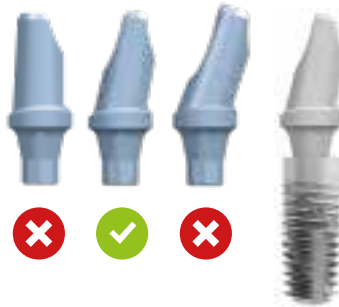
Choose the EASY LAB Replica with the angle corresponding to the EASY Abutment used in the clinic

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Screw the EASY LAB Replica onto the GTB analogue with the dedicated retaining screw

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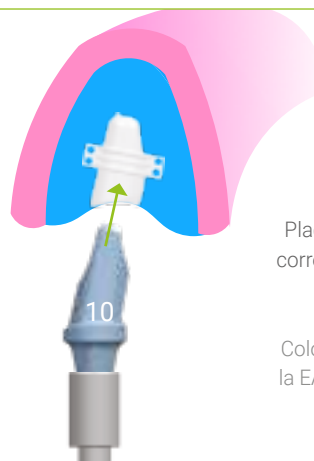
Place the assembled component in the corresponding EASY Skin Cap and create the master model

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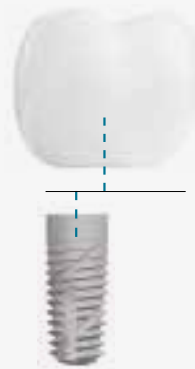
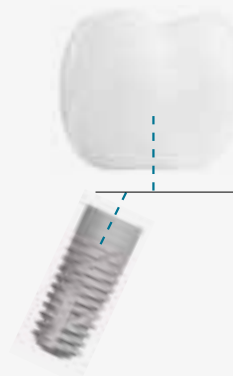
**MUA**

The MUA has been accepted as a benchmark for the rehabilitation of edentulous arches or screwed prosthesis

The MUA is carefully designed to fit edentulous and partially edentulous arches, especially when using the concept of Zero Cantilever treatment with screwed prosthesis as documented in the literature [12][15].

**Accurate design:**

Short cone for limited interclosure space. Large shoulder to facilitate placement of prosthetic reconstruction. For the various anatomical configurations of soft tissues it is available straight and angled (at 17° and 30°).

**Handy and safe:**

Each MUA comes with an exclusive pre-mounted support that facilitates the positioning. It also serves as a visual guide for checking the correct angle of abutment.

**Aesthetics designed with precision:**

Allows to work with the most modern CAD / CAM systems for implanted artifacts, impeccable both mechanically and aesthetically.

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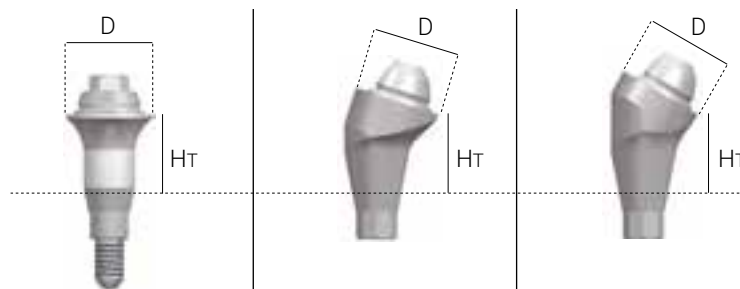
**MUA**

Multiprotesic system with screwed connection. For screwed bridges, screwed retentive bars, Toronto-Bridge, totally edentulous arch rehabs. Ideal for rehabilitation with more than 3.0mm of gingiva. Compatible with the most popular standard on the market. Available straight and pre angled at 17° and 30°. Packed with carrier. Sterile packaging. Made in Ti gr.23.

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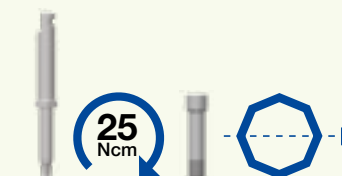
**05GND05**

D = 4.8mm  
HT = 5.5mm  
∠ = 0°


Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE A included  
Angled respect octagon vertex  
Mount pre-assembled included

**05GND06**

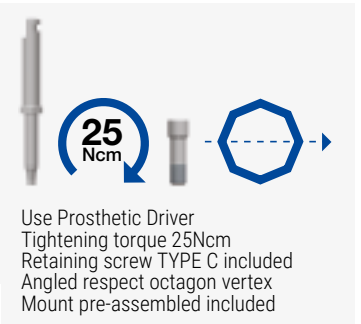D = 4.8mm  
HT = 6.5mm  
∠ = 0°

Use MUA Driver  
Tightening torque 25Ncm  
Mount pre-assembled included

Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE C included  
Angled respect octagon vertex  
Mount pre-assembled included

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**MUA PROSTHETIC COMPONENTS**

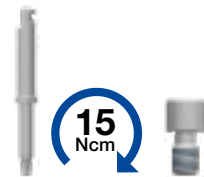
The MUA system has a series of prosthetic components that allow to condition soft tissues, make an impression with open tray technique, create a master model with a dedicated analog and realize temporary and definitive prosthetic rehabilitation.

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**01CMU10**  
MUA CASTABLE SLEEVE

Made in PMMA.  
Use Prosthetic Driver.  
Tightening torque 15 Ncm.  
Retaining screw TYPE D included.

**01TMU10**  
MUA IMPRESSION POST

**01AMU10**  
MUA ANALOG

**01SMU10**  
MUA SCAN ABUTMENT  
Retaining screw TYPE D included

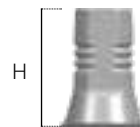
Made in Ti gr 23.  
Use Prosthetic Driver.  
Tightening torque 7 Ncm.

**01CMU20**  
MUA TITANIUM SLEEVE


**01GMU10**  
MUA HEALING CUP

**01BMU10**  
MUA UNI-BASE  
H = 7.0 mm

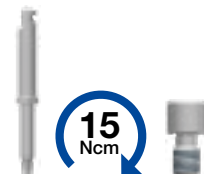**01BMU20**  
MUA UNI-BASE  
H = 9.0 mm

Made in Ti gr 23.  
Use Prosthetic Driver.  
Tightening torque 15 Ncm.  
Retaining screw TYPE D included.

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## MUA/GFA COUNTERSINK AND STOPS

The GTB system provides marginal bone preparators designed on the exact emerging profile of the MUA/GFA abutments so that it can be easily used the sub-crestal positioning approach of the implant and always be sure of the correct positioning of the MUA/GFA on the implant itself. This benefits the ease and speed of work and avoids radiographic controls only to check the correct position of the prosthetic abutment. The stops are easily screwed onto the implant and serve as a guide for the preparator and stopping it to the correct height (available for 3,5-4,5-5,5-6,5 mm transmucosus). Diamond Like Carbon Surface Finish.

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
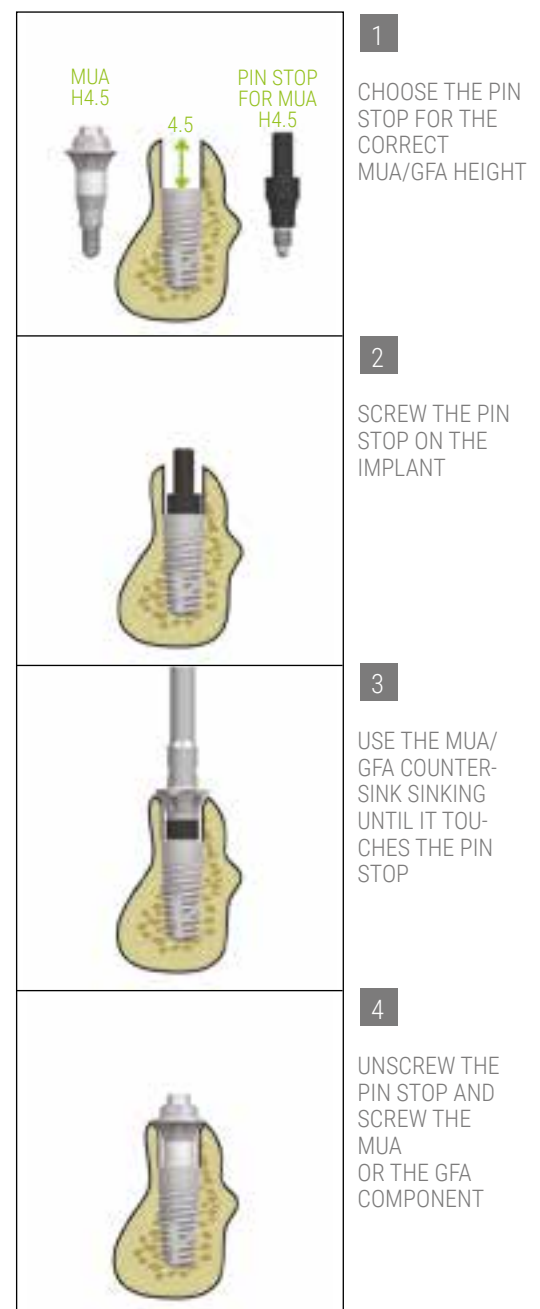
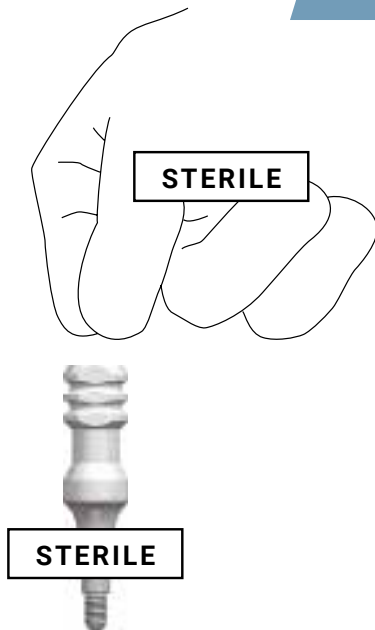
Take the MUA using the carrier and transport it onto the implant

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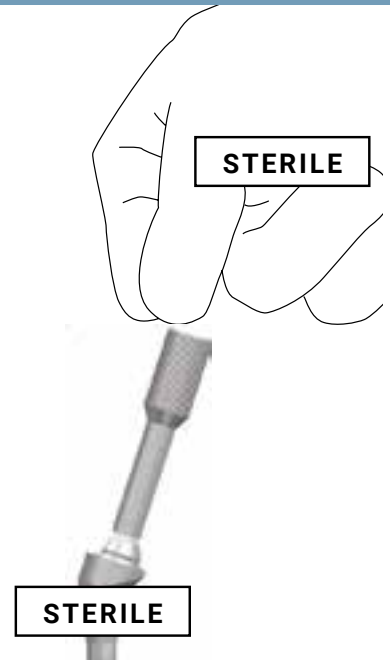Screw the straight MUA manually using the carrier

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Choose the correct position of the angled MUA using the carrier

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Remove the carrier by bending it sideways

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Tighten the retaining screw with the Prosthetic Driver to 25 Ncm

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Use the MUA Driver to tighten the straight MUA to 25 Ncm

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The GFA system consists of two parts:

- 1 - a primary component called Gingival Former
- 2 - a set of various components to be used according to the type of prosthetic rehabilitation you want to accomplish.

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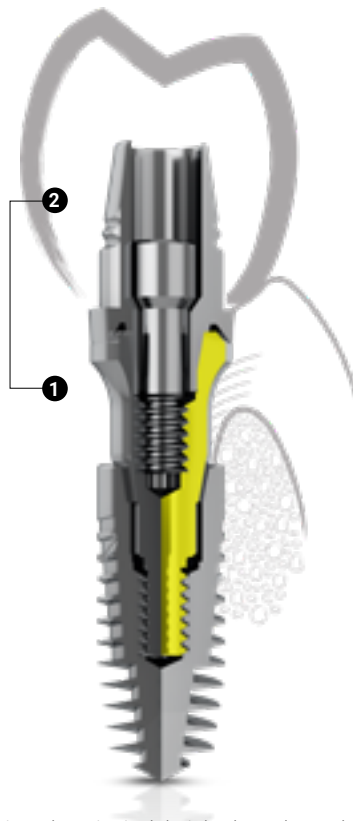
## GFA

The GFA system is a prosthesis system for dental implants GTB that conceptually allows a submerged implant, crestal or subcrestal, to be used as a transmucosal implant. The GFA primary component is no longer removed during prosthetic rehabilitation (if necessary, it is still possible to remove it easily). It can be used both as a healing abutment and as a new prosthetic platform for secondary prosthetic components. This prevents the formation of a deep epithelial attack around the healing abutment and then around the prosthetic abutment. How is this happening? The connection present in the implant is replicated and positioned at prosthetic platform level of the GFA primary component. So you can choose to elevate towards a coronal position the same connection present at implant platform, bringing the same implant-abutment connection at tissue level, facilitating future prosthetic procedures.

Choosing the gingival height based on the available biological width, you can position the correct GFA primary component then leave it in place during all the prosthetic steps to be performed later, avoiding the formation of a deep epithelial attack and consequently avoiding the risk factor given by the formation of a periimplant pocket due to, in conventional treatments, continuous positioning and removal of healing and prosthetic components [1][14][23][27].

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Also available:

- a dedicated pick-up transfer
- a dedicated laboratory analog
- a dedicated scan body for optical scanners

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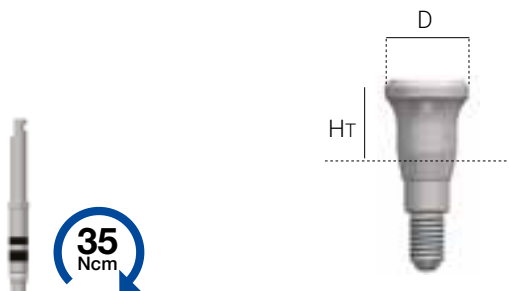
### GINGIVAL FORMER

Multiprotesic system with screwed conical connection. The GFA system allows to select a transosseus and/or transgingival primary component between 5 different heights, called Gingival Former. The secondary connection, present in the coronal position of the Gingival Former, allows to perform any type of prosthetic rehabilitation: from the single crown to the full arch, from the cemented prosthesis to the screwed one, from the retention bar to the fixed full arch, from CAD/CAM rehabilitation to the castable abutment. It is a new prosthetic platform with the same features of the GTB implant connection. Packaged with carrier. Sterile packaging. Made in Ti gr.23.

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Use Implant Driver  
Tightening torque 35Ncm  
Mount pre-assembled  
included

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### GFA COVER SCREWS

Cover Screws for Gingival Former. Available in three heights. They make the Gingival Former a traditional healing abutment. After the healing period, only the cover screw will be removed leaving the Gingival Former in place. The cover screws do not engage the conical connection avoiding accidental loosening of the Gingival Former. Made in Ti gr.23.

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### GFA REPLICA

Analog replicating the Gingival Former connection. Made in Ti gr.23.

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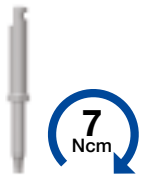
**GFA IMPRESSION POST**

Gingival Former impression post for open tray technique. Made in Ti gr.23.

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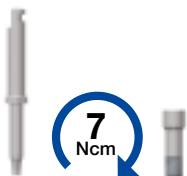
**GFA SCAN ABUTMENT**

Gingival Former abutment for digital impression. Made in Ti gr.23 sandblasted.

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Use Prosthetic Driver  
Tightening torque 7Ncm  
Retaining screw TYPE C  
included

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**GFA TEMPORARY SLEEVE**

Gingival Former temporary sleeve for screw retained and cement retained crowns. Available in not locking and locking versions. Made in Ti gr.23.

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			 <p>Use Prosthetic Driver Tightening torque 25Ncm Retaining screw TYPE C included</p>
<b>01CGF20</b> LA = 12.0mm LOCKING	<b>01CGF22</b> LA = 12.0mm NOT LOCKING	<b>01CGF23</b> LA = 12.0mm LOCKING	

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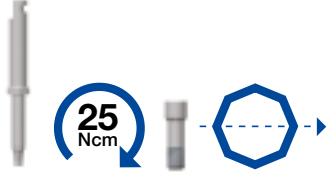
**GFA ABUTMENTS**

Customizable abutments for Gingival Former. Pre-formed and pre-angled, can be milled. They allow to shape knife-edge crowns or to recreate a new crown finishing margin by chamfer or bevel. Available in locking version. Made in Ti gr.23.

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### GFA OVER-CASTING ABUTMENT WITH CoCr28Mo BASE

Locking over-casting abutment for Gingival Former. Base made in CoCr28Mo and castable sleeve made in POM.

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Use Prosthetic Driver  
Tightening torque 25Ncm  
Retaining screw TYPE C  
included

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### GFA PRE-MILLED ABUTMENT

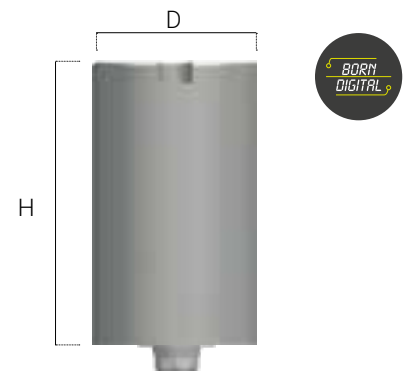
This prosthetic component has a pre-machined connection with all the characteristics and all the tight tolerances guaranteed by the Advan implant system. The Pre-Milled abutment allows to obtain a customized abutment, regardless of the Gingival Former position. Made in CoCr28Mo.

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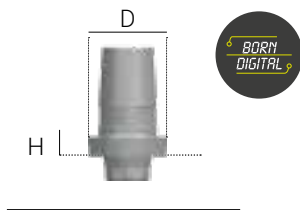
**GFA TI-BASE ABUTMENT**

Ti-Base is the titanium base for CAD/CAM rehabilitation. Compatible with Cerec Standard S by Dentsply Sirona. Made in Ti gr.23

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**41GLC06**  
 D = 4.1 mm  
 H = 1.0 mm  
 LOCKING

Use Prosthetic Driver  
 Tightening torque 25Ncm  
 Retaining screw TYPE C included

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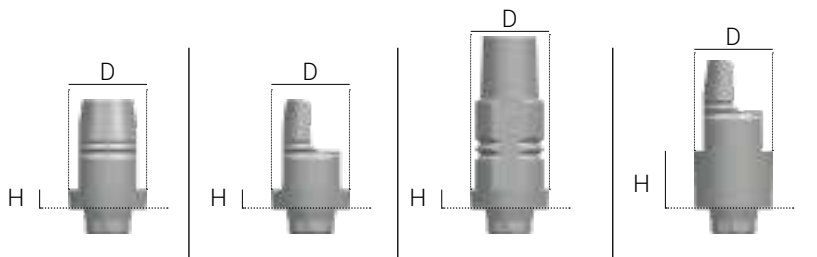
**GFA UNI-BASE ABUTMENT**

Uni-Base is the universal titanium base for CAD/CAM rehabilitations. Available in different heights and sleeve shapes to adapt to every prosthetic need. Made in Ti gr.23

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### MUA/GFA COUNTERSINK AND STOPS

The GTB system provides marginal bone preparators designed on the exact emerging profile of the MUA abutments so that it can be easily used the sub-crestal positioning approach of the implant and always be sure of the correct positioning of the MUA on the implant itself. This benefits the ease and speed of work and avoids radiographic controls only to check the correct position of the prosthetic abutment. The stops are easily screwed onto the implant and serve as a guide for the preparator, stopping it to the correct height (available for 3.5-4.5-5.5-6.5 mm transmucosus). Diamond Like Carbon Surface Finish.

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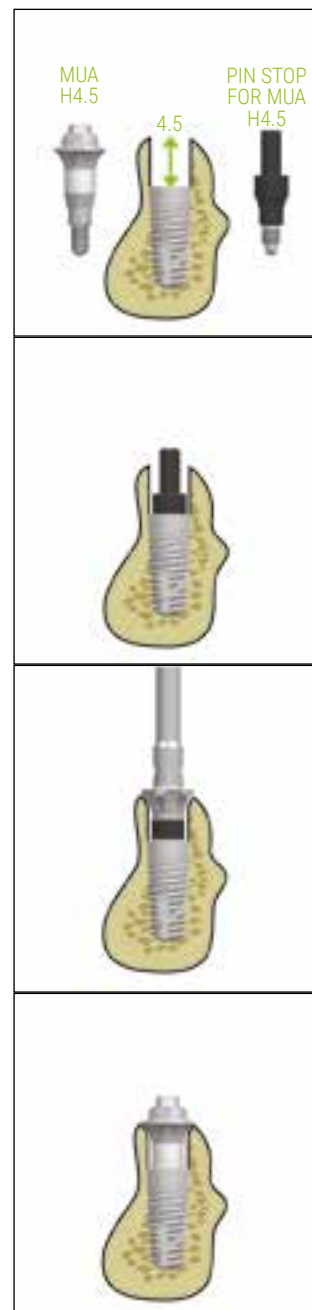
#### GKM10

MUA/GFA COUNTERSINK DRILL SET

MUA/GFA COUNTERSINK  
PIN STOP MUA/GFA H3.5mm  
PIN STOP MUA/GFA H4.5mm  
PIN STOP MUA/GFA H5.5mm  
PIN STOP MUA/GFA H6.5mm

Use Prosthetic Driver  
Tightening torque 7 Ncm

1

CHOOSE THE  
PIN STOP FOR  
THE CORRECT  
MUA/GFA  
HEIGHT

2

SCREW THE PIN  
STOP ON THE  
IMPLANT

3

USE THE MUA/  
GFA COUNTER-  
SINK UNTIL IT  
TOUCHES THE  
PIN STOP

4


UNSCREW THE  
PIN STOP AND  
SCREW THE  
MUA  
OR THE GFA  
COMPONENT

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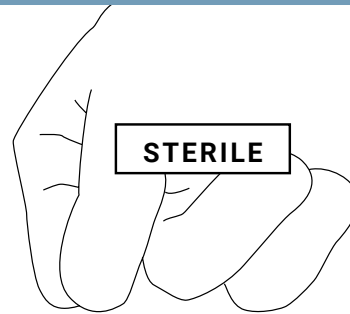Take the Gingival Former using the carrier and transport it onto the implant

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Screw the Gingival Former manually using the carrier

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Use the Implant Driver to tighten the Gingival Former to 35 Ncm

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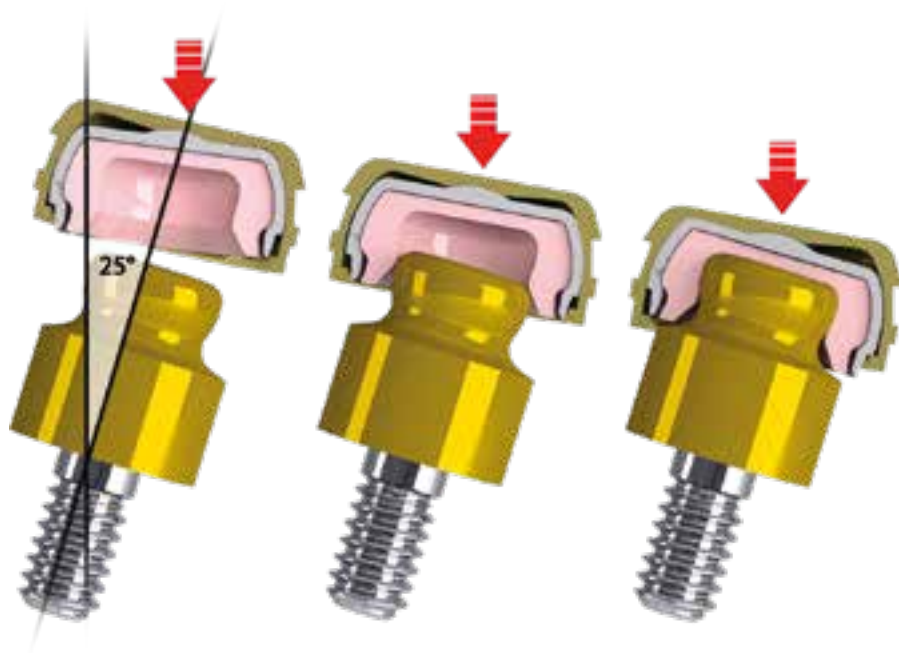
# RETENTIVE PROSTHETIC MULTISYSTEM

## OT EQUATOR

## Smart BOX

50° DIVERGENCE CORRECTION

OT Equator retentive prosthetic multisystem,  
spare parts, accessories and specific tools  
are made by 

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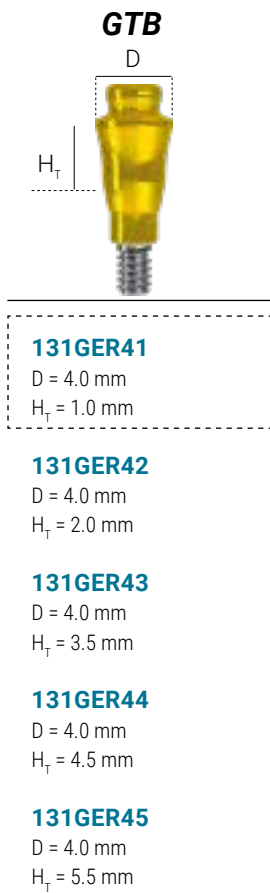
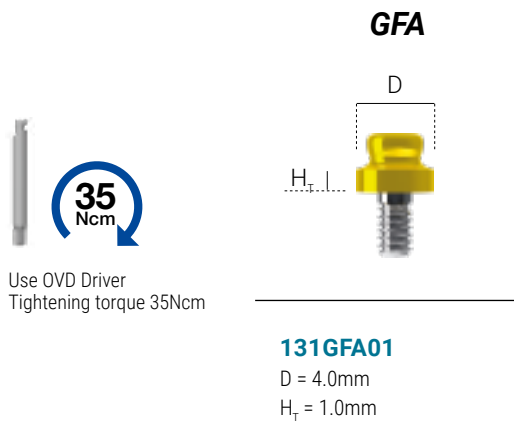
**OT EQUATOR RETENTIVE PROSTHETIC MULTISYSTEM**

The Ot Equator® overdenture abutments guarantee an excellent retention with minimum vertical and horizontal dimensions. Made of Ti gr.5 and covered TiN. The SmartBOX case for caps allow to correct up to 50° of disparallelism between the implants, guaranteeing a safe and comfortable positioning of the prosthesis.

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- Each Ot Equator® SmartBox package contains:
- 1 Ot Equator® abutment
  - 1 SmartBox case
  - 1 protective disk for relining
  - 4 assorted retentive cap (1 extra-soft, 1 soft, 1 standard, 1 strong)
  - 1 protective cap

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**PROSTHETIC SCREW TYPE A**

Prosthetic screw for components. Made in Ti gr.23.

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**DYNAMIC SCREW TYPE A**

Dynamic screw for prosthetic components. Made in Ti gr.23.

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### PROSTHETIC PLANNING KIT

Replicas of the prosthetic abutments to quickly choose the correct transmucosal height and angle.  
Sterilizable replicas by autoclave.

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**Magellan** is ADVAN's guided surgery system that allows the management of the clinical case at 360 degrees, from software planning to the final intervention.

Magellan software allows to perform three-dimensional implant simulation directly on the Personal Computer. It also allows to simulate the position of the implants on two-dimensional and three-dimensional models, identify the mandibular canal, trace panoramas and sections of the bone model, also offering the possibility of calculating bone density. Using Magellan, the dentist can plan the implant-prosthetic surgery in a safer, more efficient and faster way.

The surgical kit consists of a single box containing all the surgical and prosthetic instruments necessary for the realization of the osteotomy, the positioning of the implant and its prosthesis. Including surgical box made of Radel (non-toxic, autoclavable, metal-free).

The content of the GTB guided surgery kit was developed by ADVAN to support the complete workflow in guided surgery, from preparation of the implant site to final implant placement. All the tools and drills in the kit are used with a single  $\varnothing 4.5$ mm bushing without the use of stops or keys to be assembled to the components at a later time, allowing to reduce the surgery time to a minimum.

The drills of the Magellan kit allow the use of all implant diameters and all lengths, with the exception of the 15.0mm and 18.0mm length.

Furthermore, to allow the correct insertion of the GTB implants in the subcrestal position, special implant mounts were made, oversized in length by 1.5mm.

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### GUIDED SURGERY MAGELLAN KIT

The surgical kit consists of a single box containing all the surgical and prosthetic instruments necessary for the realization of the osteotomy and the implant positioning using the surgical guide. The kit contains the drivers necessary to implant prosthesis. Including Radel made surgical tray (non-toxic, autoclavable, metal-free).

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**KCG10** GTB GUIDED SURGERY KIT

<b>01BCG45</b>	GTB Guided Surgery Sockets (5 pcs.)	1	<b>30FCG09</b>	Compact bone drill Ø3,0x9,0	1
<b>02-AC50</b>	Manual adapter	1	<b>30FCG10</b>	Compact bone drill Ø3,0x10,5	1
<b>FS14</b>	Lateral Pin	2	<b>30FCG12</b>	Compact bone drill Ø3,0x12,0	1
<b>FS15</b>	Lateral Pin drill	1	<b>30FCG13</b>	Compact bone drill Ø3,0x13,5	1
<b>07-EG10</b>	Prosthetic driver	1	<b>33FCG06</b>	Compact bone drill Ø3,3x6,0	1
<b>07MCG10</b>	Mucotome	1	<b>33FCG07</b>	Compact bone drill Ø3,3x7,5	1
<b>07DMG10</b>	Mount driver	1	<b>33FCG09</b>	Compact bone drill Ø3,3x9,0	1
<b>02ECG10</b>	Manual extractor	1	<b>33FCG10</b>	Compact bone drill Ø3,3x10,5	1
<b>25FCG06</b>	Alveolar bone drill Ø2,5x6,0	1	<b>33FCG12</b>	Compact bone drill Ø3,3x12,0	1
<b>25FCG07</b>	Alveolar bone drill Ø2,5x7,5	1	<b>33FCG13</b>	Compact bone drill Ø3,3x13,5	1
<b>25FCG09</b>	Alveolar bone drill Ø2,5x9,0	1	<b>40FCG06</b>	Compact bone drill Ø4,0x6,0	1
<b>25FCG10</b>	Alveolar bone drill Ø2,5x10,5	1	<b>40FCG07</b>	Compact bone drill Ø4,0x7,5	1
<b>25FCG12</b>	Alveolar bone drill Ø2,5x12,0	1	<b>40FCG09</b>	Compact bone drill Ø4,0x9,0	1
<b>25FCG13</b>	Alveolar bone drill Ø2,5x13,5	1	<b>40FCG10</b>	Compact bone drill Ø4,0x10,5	1
<b>28FCG06</b>	Alveolar bone drill Ø2,8x6	1	<b>40FCG12</b>	Compact bone drill Ø4,0x12,0	1
<b>28FCG07</b>	Alveolar bone drill Ø2,8x7,5	1	<b>40FCG13</b>	Compact bone drill Ø4,0x13,5	1
<b>28FCG09</b>	Alveolar bone drill Ø2,8x9,0	1	<b>07FGM32</b>	Marginal bone drill Ø3,2	1
<b>28FCG10</b>	Alveolar bone drill Ø2,8x10,5	1	<b>07FGM35</b>	Marginal bone drill Ø3,5	1
<b>28FCG12</b>	Alveolar bone drill Ø2,8x12,0	1	<b>07FGM42</b>	Marginal bone drill Ø4,2	1
<b>28FCG13</b>	Alveolar bone drill Ø2,8x13,5	1	<b>07FPC10</b>	Crestal Pin drill	1
<b>36FCG06</b>	Alveolar bone drill Ø3,6x6,0	1	<b>KC0103</b>	Box	1
<b>36FCG07</b>	Alveolar bone drill Ø3,6x7,5	1	<b>01MCG01</b>	Implant mount h0 (2 pcs.)	2
<b>36FCG09</b>	Alveolar bone drill Ø3,6x9,0	1	<b>01MCG02</b>	Implant mount h1,5 (2 pcs.)	2
<b>36FCG10</b>	Alveolar bone drill Ø3,6x10,5	1	<b>07PLG10</b>	Planisher	1
<b>36FCG12</b>	Alveolar bone drill Ø3,6x12,0	1	<b>01PCG10</b>	Crestal Pin	2
<b>36FCG13</b>	Alveolar bone drill Ø3,6x13,5	1	<b>01VMG16</b>	Mount retaining screw (2 pcs.)	2
<b>30FCG06</b>	Compact bone drill Ø3,0x6,0	1	<b>FN02</b>	GTB Guided surgery Sockets for lateral PIN (3 pcs.)	1
<b>30FCG07</b>	Compact bone drill Ø3,0x7,5	1			

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### ALVEOLAR BONE DRILLS MAGELLAN

The alveolar bone drills have a single ring color code for easy recognition of the diameter. They have a small apex diameter to create the correct implant preparation with soft and cancellous bone. The drills create an osteotomy that can be considered congruous to the length of the chosen implant since the maximum increase in depth is only 0.4 mm (due to the conical apex of the drill unlike the convex apex of the implant). External irrigation cutters.

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### COMPACT BONE DRILLS MAGELLAN

The compact bone drills are color-coded with a double ring for easy recognition of the diameter. They have a cylindrical profile to create the correct implant preparation with compact bone. The drills create an osteotomy that can be considered congruous to the length of the chosen implant since the maximum increase in depth is only 0.4 mm (due to the conical apex of the drill unlike the convex apex of the implant).

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#### 30FCG00 SET COMPACT BONE DRILLS Ø3,0mm

<b>30FCG06</b>	Compact bone drill Ø3,0x6,0	1
<b>30FCG07</b>	Compact bone drill Ø3,0x7,5	1
<b>30FCG09</b>	Compact bone drill Ø3,0x9,0	1
<b>30FCG10</b>	Compact bone drill Ø3,0x10,5	1
<b>30FCG12</b>	Compact bone drill Ø3,0x12,0	1
<b>30FCG13</b>	Compact bone drill Ø3,0x13,5	1

#### 33FCG00 SET COMPACT BONE DRILLS Ø3,3mm

<b>33FCG06</b>	Compact bone drill Ø3,3x6,0	1
<b>33FCG07</b>	Compact bone drill Ø3,3x7,5	1
<b>33FCG09</b>	Compact bone drill Ø3,3x9,0	1
<b>33FCG10</b>	Compact bone drill Ø3,3x10,5	1
<b>33FCG12</b>	Compact bone drill Ø3,3x12,0	1
<b>33FCG13</b>	Compact bone drill Ø3,3x13,5	1

#### 40FCG00 SET COMPACT BONE DRILLS Ø4,0mm

<b>40FCG06</b>	Compact bone drill Ø4,0x6,0	1
<b>40FCG07</b>	Compact bone drill Ø4,0x7,5	1
<b>40FCG09</b>	Compact bone drill Ø4,0x9,0	1
<b>40FCG10</b>	Compact bone drill Ø4,0x10,5	1
<b>40FCG12</b>	Compact bone drill Ø4,0x12,0	1
<b>40FCG13</b>	Compact bone drill Ø4,0x13,5	1

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### MARGINAL BONE DRILLS MAGELLAN

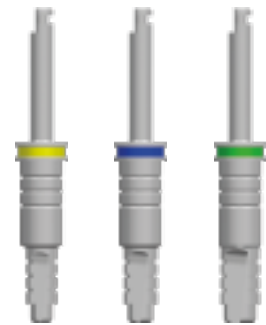
The marginal bone drills have a single color code ring to easily recognize the diameter. The marginal bone drills have a self-centering apex. Milled work depth references. External irrigation drills.

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### INSTRUMENTS AND MOUNTS MAGELLAN

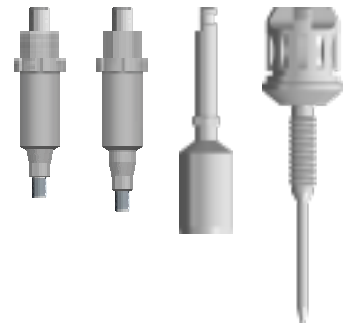
The implant mounts allow the correct positioning of the implant in the bone both in the crestal and sub-crestal position, thanks to special implant mounts, oversized in length of 1.5mm. The kit also contains the drivers for the mounts and the convenient extractor in case of difficulties in removing the mount from the implant.

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**OPTIONAL DRILLS & PINS MAGELLAN**

The kit of optional drills includes the mucotome, to remove the first layer of gingiva and prepare the site for the osteotomy, the initial drill and drills for the insertion of the crestal and lateral pins.

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The aim of **Advan** is to deliver to the treatment Providers products that guarantee safety, efficacy and allow professional efficiency. During our activities for product development and product improvement **Advan** reserves the right to make changes to the products, instruments, surgical techniques and Instructions for use (IFU) if it constitutes a valuable improvement for the treatment Providers. **Advan** reserves the right to modify prices, delivery terms and sales terms without notice. Products might not be available in all Countries.

Lifetime Warranty: all implants and prosthetic components of **Advan** include a lifetime warranty that guarantees to replace any product that fails both for materials or manufacturing defects. This warranty is valid only if products are used in conjunction with the associated original components and instruments, used according to the Instructions for Use (IFU). The warranties given are exclusive for the benefit of eligible treatment Providers and are not for any other person or entity, including any patient.

**Advan** replaces the defective products according to the product defect and corrective action requested. The treatment Provider assumes all risks and liabilities deriving from the use of product, the lifetime warranty is limited to the replacement of defective product and **Advan** disclaims any liability to the treatment Provider or Patient and does not compensate costs for any additional components, instruments, treatment costs and any other cost related to or originated from the replacement of defective product.

Treatment planning and clinical application of the **Advan** products are under exclusive responsibility of treatment Provider; **Advan** recommends to attend intensive training programs, to complete a postgraduate dental implant courses and to strict adherence the Instructions for Use (IFU) and any other indication contained in the technical documentation available to Users. The product sale is authorized only to treatment Providers according with local regulatory rules and Laws. The products of this catalog could be patented or patent pending.

Any replacement of product can be authorized by **Advan** only after submission of the "Guarantee Questionnaire" available upon request from Customer Service Office.

The "Guarantee Questionnaire" form must be filled in all the parts, duly signed and stamped, accompanied by the failed product and the other components placed by the treatment Provider within ONE (1) month after the product failure occurred. In case of an implant failure, x-ray of the failed implant IS MANDATORY. Prior to submit implant and the other components used in the treatment, ensure that sterilization of all the product has been made.

Warning – read carefully instructions for use (IFU) before product use

#### LIMITS AND LIMITATIONS

**ADVAN** HEREBY DISCLAIMS ANY OTHER WARRANTIES, EXPRESS OR IMPLIED AND **ADVAN** HEREBY EXCLUDES ANY LIABILITY FOR LOST EARNINGS AND DIRECT OR INDIRECT DAMAGES AS WELL AS COLLATERAL AND CONSEQUENTIAL DAMAGES, DIRECTLY OR INDIRECTLY RELATED TO **ADVAN** PRODUCTS, SERVICES OR INFORMATION.

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	Manufacturer		Do not re-use
	Date of manufacture		Consult instructions for use or consult electronic instructions for use
	Use-by date		Caution
	Batch code		Medical device
	Catalogue number		Unique device identifier
	Distributor		Advan products covered by the CE mark fulfill the requirements of the Directive 93/42/CEE concerning medical devices and falls within Classes IIa, IIb
	Sterilized using irradiation		Advan products covered by the CE mark without the identification number fulfill the requirements of the EU Regulation 2017/745 (MDR) concerning medical devices and falls within Class I
	Do not re-sterilize		Not locking prosthetic component
	Non-sterile		Octagon locking prosthetic component
	Do not use if package is damaged and consult instructions for use		Multi packaging (the number reported in the symbol refers to the number of units in the packaging)
	Single sterile barrier system with protective packaging inside		
	Single sterile barrier system with protective packaging outside		
	Keep away from sunlight		
	Keep dry		

ADVAN

BORN  
DIGITAL

*NATIVE DIGITAL WORKFLOW INTEGRATION*

WWW.ADVANIMPLANTOLOGY.COM

Advan Implantology  
Via Rosta della Maina, 2  
33020 Amaro (UD)  
Tel. (+39) 0433.096245  
email: info@advanimplantology.com

FOLLOW US